# HOMEDICS®

#1 BRAND IN MASSAGE\*

PLEASE TAKE A  
**MOMENT NOW**  
REGISTER YOUR PRODUCT AT:

[www.homedics.com/register](http://www.homedics.com/register)

Your valuable input regarding this product will help us create the products you will want in the future.

# SERENITY SHIATSU MASSAGE CUSHION

WITH SOUND & MEDITATION

INSTRUCTION MANUAL AND WARRANTY INFORMATION

**2-YEAR LIMITED WARRANTY**

MCS-1005HJ

# IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

## DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- NEVER use pins or other metallic fasteners with this appliance.

## WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically, any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to a HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the off position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- This appliance is designed for personal, nonprofessional use.
- DO NOT use this massager in close proximity to loose clothing or jewelry.
- Keep long hair away from massager while in use.

**CAUTION: All servicing of this massager must be performed by authorized HoMedics service personnel only.**

## SAVE THESE INSTRUCTIONS

CAUTION – PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- Consult your doctor prior to using this product, if:
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- Not recommended for use by diabetics.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bedtime. The massage has a stimulating effect and can delay sleep.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This appliance is intended for household use only.

# INSTRUCTIONS FOR USE

The cushion comes with a screw located on the back of the unit to protect the massage mechanism during shipment, which must be removed before the first use to allow the unit to function properly. Use the provided Allen wrench. Then, properly dispose of the screw.

**WARNING: Failure to remove screw may cause permanent damage to the massage cushion. DO NOT REINSTALL SCREW.**

To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. Untie cord strap, extend cord, and insert plug into a 120-volt household electrical outlet to power unit.
2. Attach the cushion and enjoy your massage as you read, rest, or even work. Fits most chairs.

See pages 8-9 for instructions on the use of the hand control.

## SOUND AND MEDITATION

Enhance your massage with a variety of soothing sounds and meditations. Smart device users can go to the Apple App Store or the Google Play Store to download our HoMedics Massage App. See page 7 for more instructions on the app.

## REMOVABLE COVER

The cushion includes a removable cover that, when used, will customize the intensity of your massage. The cushion may be operated with or without the removable cover. Just use the included hook and loop fastener at the top of the cover to attach or remove the cover from the cushion.

## STRAPPING SYSTEM

The cushion incorporates a unique strapping system that allows you to fasten it to almost any chair. Just slip the straps over the seat or chair and adjust fastening straps to secure. Your massager won't slip or slide away.

---

**CAUTION: To avoid pinching, do not lean on the back massage mechanism in the cushion when adjusting your body position.**

**DO NOT jam or force any part of your body in the moving massage mechanism.**

**NOTE:** This unit is equipped with a 15-minute auto-off feature for your safety.

# MAINTENANCE

## TO STORE

Place cushion in its box or in a cool, dry place. Avoid contact with sharp edges or pointed objects that might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit.

## TO CLEAN

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- DO NOT immerse in any liquid to clean.
- NEVER use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean.
- DO NOT attempt to repair the massage cushion. There are no user-serviceable parts. For service, call Consumer Relations at the number listed in the Warranty section.

---

**NOTE:** Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using on upholstered furniture.

**NOTE:** Only gentle force should be exerted against the unit in order to eliminate any/all risk of injury. You may soften massage force by placing a towel between yourself and the unit.

# FCC STATEMENT

This device complies with Part 15 of the FCC Rules and Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:** The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and CAN ICES-3 (B)/NMB-3 (B). These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## FCC & IC RADIATION EXPOSURE STATEMENT:

This equipment complies with FCC and Canada radiation exposure limits set forth for an uncontrolled environment.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

# MESSAGE APP QUICK-START GUIDE

1. From your smartphone or tablet, access the appropriate online store.


Download on your iPhone® or iPad®.


Download on your Android™ device.

2. Search for “HoMedics Massage.”
3. Install the HoMedics Massage app on your device.
4. Ensure your device’s Bluetooth® capability option is turned on.
5. Ensure your HoMedics product is powered on.
6. Open the app, and tap the + icon in the top right of your screen to add a device.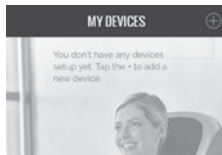

7. Choose the product you'd like to pair with your mobile device from the drop down menus, and tap the + icon to pair.

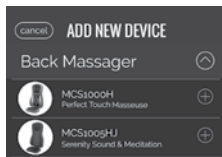8. Your device will pair to your selected HoMedics product automatically.
9. You are now able to control your HoMedics massage product using your smartphone or tablet and enjoy additional product features. For more detailed instructions about app functionality, please visit <http://www.homedics.com/instructionbooks/> and select HOMEDICS-MASSAGE-APP from the drop down list.

## POWER

To turn on the massage functions, first press . The LED indicator will illuminate.

## BLUETOOTH INDICATOR

When the massage cushion is powered on, the Bluetooth® indicator will flash blue while pairing with your mobile device. Download the HoMedics Massage App and follow the pairing instructions. When your mobile device is successfully paired with the cushion, the indicator will have a steady blue LED to indicate it is paired.

**NOTE:** The blue LED will flash for 30 seconds when attempting to pair. If it does not successfully pair with your device, it will turn off.

## NECK AND SHOULDER MASSAGE

To turn on neck and shoulder massage, press the mode button and the LED will illuminate. To turn off, press the mode button a second time.

## BACK MASSAGE ZONES

To select a zone, press the lower, upper, or full back massage buttons and the LED will illuminate. To deselect, press the button again or select a different zone.

## VIBRATION

Press the vibration button to enjoy vibration massage in the seat cushion.

**NOTE:** When the power button is pressed for first-time use, the moving mechanism will automatically move to the lowest position before starting a massage program. After initial use, the mechanism will always park at the lowest position. See note below regarding parking.

**NOTE:** The moving massage mechanism always “parks” or finishes in its lowest position. It continues to this position after the power has been turned off. If the electrical supply is interrupted, when power is restored, the mechanism will “park” or move to its lowest position.

## DEMO

For a brief demonstration of the features of the massage cushion, press the DEMO button. The LED light will illuminate to indicate active function. The unit will briefly go through each function. Once completed, the unit will automatically turn off.

## MODE

Choose from rolling, shiatsu, tapping, or shiatsu and tapping combined. Press the MODE button and the corresponding LED on the selected massage style will illuminate.

### ROLLING MESSAGE

Rolling massage simulates 2 hands pressing firmly and slowly traveling up and down your back.

### SHIATSU MESSAGE

Shiatsu is a deep-kneading massage. During this function, the massage nodes will rotate while traveling up and down your back, soothing tense muscles.

### TAPPING MESSAGE

Tapping is a fast, invigorating massage.

## NECK AND BACK SPOT MASSAGE

While the unit is in shiatsu mode, press either SPOT button to stop and focus the shiatsu action in one place. Once the unit is operating in spot massage, you may adjust the mechanism position by holding down either button until the desired position is reached. The up arrow will move the mechanism upward and the down arrow will move it downward. Once you release the button, the mechanism will stop in that place.

## HEAT

This massage cushion is equipped with heat in the neck, back, and seat cushion. For soothing heat, when massage is on, press the HEAT button and the corresponding red LED light will illuminate. To deselect, press the button again and the corresponding LED will turn off. Heat will only work when the massage feature is on.

#1 BRAND IN MESSAGE\*

## 2-YEAR LIMITED WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including but not limited to Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adaption to enable it to operate in any other country than the country for which it was designed, manufactured, approved, and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL, OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights that may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit [www.homedics.com](http://www.homedics.com). For Canada, please visit [www.homedics.ca](http://www.homedics.ca).

---

### FOR SERVICE IN THE USA

[cservice@homedics.com](mailto:cservice@homedics.com)

8:30am-7:00pm EST Monday-Friday

1-800-466-3342

### FOR SERVICE IN CANADA

[cservice@homedicsgroup.ca](mailto:cservice@homedicsgroup.ca)

8:30am-5:00pm EST Monday-Friday

1-888-225-7378

\*Source: The NPD Group, January 2018-December 2018, Retail Tracking Service.

©2019 HoMedics, LLC. All rights reserved.

HoMedics and HoMedics #1 Brand In Massage are registered trademarks of HoMedics, LLC.

Distributed by HoMedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390

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©2019 HoMedics, LLC. Todos los derechos reservados.
HoMedics y HoMedics #1 Brand In Massage es una marca registrada de HoMedics, LLC.
Distribuido por HoMedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390