HOMEDICS[®]

#1 BRAND IN MASSAGE*

ENGLISH
EN ESPAÑOL, PG 9

SPORTS RECOVERY

MERCURY[™] PERCUSSION MASSAGER WITH **HEAT**

INSTRUCTION BOOKLET AND SAFETY INFORMATION

SR-PRCM

IMPORTANT SAFEGUARDS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.

WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE, OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.
- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Do not use massager in close proximity to loose clothing or jewelry. Keep long hair away from massager while in use.

SAVE THESE INSTRUCTIONS

CAUTION—PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- Not recommended for use by people with diabetes.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product’s excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user’s capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.
- NEVER use this product in automobiles.
- This appliance is intended for household use only.
- **Attention California Residents:**
WARNING: This product contains chemicals known to the State of California to cause cancer, birth defects, and other reproductive harm.

**Power Switch/
Variable Intensity Control**
For gentle to invigorating
massage at the turn of
a dial


Comfortable Grip
Fits easily in the
palm of your hand

Adjustable Strap
Fits your hand size

Thermal Heat

**Powerful
Percussion Action**
For an invigorating
massage

Lightweight Design
Small enough to
carry wherever
you go

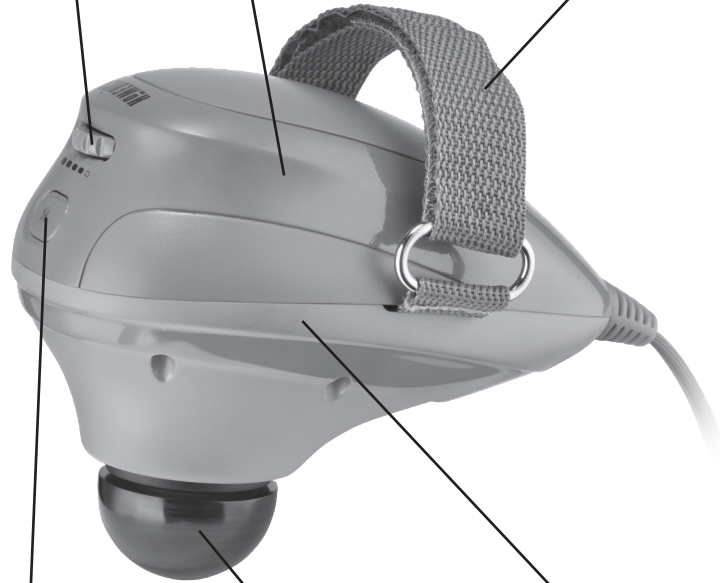INSTRUCTIONS FOR USE

To reduce the risk of electric shock, this appliance has a polarized plug (one blade wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. DO NOT change the plug in anyway.

1. ALWAYS make sure the unit is set to the “OFF” position before inserting the plug into a 120-volt electrical outlet.
2. Grip the massager by placing your hand through the strap and grasp the unit with fingers outstretched. The strap should be positioned over the back of your hand. (Fig. 1)
3. **Massage:** Slide the power switch once to the right to initiate massage. To adjust the massage intensity, rotate the switch to the right until the desired intensity is reached. The massager has 4 intensity levels. (Fig. 2)

FIG 1

Massage + Heat: Apply the red thermal heat attachment. Once the massage function is on, press the heat button directly below the power switch to activate heat (Fig. 2). The red lights will illuminate to indicate that heat is on.

FIG 2

NOTE: The heat function can only be activated when the massage function is in use.

NOTE: Do not use the heat continuously for more than 15 minutes. Do not use the heat if you have sensitive skin, poor circulation, discolored areas, swelling, burned or inflamed skin, or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.


4. Apply the massager head lightly to the area you wish to massage.
5. Move the unit slowly, holding on the targeted area. Massage the area for a short period of time and move to another area. NEVER massage the same spot longer than 3 minutes.

CAUTION – NEVER use this appliance on or near eyes or other highly sensitive areas.

6. Operation longer than 15 minutes is not recommended and may cause overheating.
7. When your massage is completed, turn the power switch all the way to the left to turn the unit off. Allow the massager to cool, unplug the power cord, and store the unit properly in a dry place.

TO CHANGE ATTACHMENTS


Your massager comes equipped with three custom massage attachments.

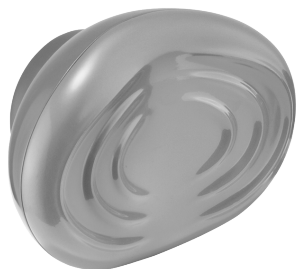A. Contoured

B. Thermal heat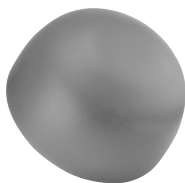

C. Pinpoint

To change to a different attachment, simply twist the attachments clockwise to add and counterclockwise to remove.

MAINTENANCE

To Clean

Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. NEVER allow water or any other liquids to come into contact with the handle.

To Store

Unplug the unit from the outlet and allow it to cool before storing in its box or a clean, dry place. Coil the cord and secure it with the supplied fastener. DO NOT hang the unit by the power cord.



Caution: All servicing of this unit must be performed by authorized HoMedics service personnel only.

FIND MORE HELP ONLINE

www.homedics.com/mercury

Visit the URL above to find how-to photos, usage tips, and additional information about this product.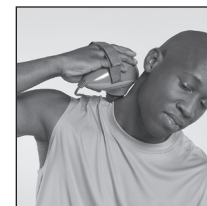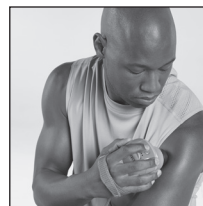

LIMITED TWO YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of two years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com. For Canada, please visit: www.homedics.ca.

CONTACT A CONSUMER RELATIONS REPRESENTATIVE

For service in the USA

Email: cservice@homedics.com
 Telephone: 1.800.466.3342
 Monday–Friday, 8:30am–7:00pm EST

For service in Canada

Email: cservice@homedicsgroup.ca
 Telephone: 1.888.225.7378
 Monday–Friday, 8:30am–5:00pm EST

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