Walgreens  
by HOMEDICS

## Manual Inflate Blood Pressure Monitor

# TABLE OF CONTENTS

## INTRODUCTORY INFORMATION:

<b>Important Product Notices and Safety Instructions</b> .....	<b>3</b>
<b>About Blood Pressure</b> .....	<b>5</b>
What is Blood Pressure?	
Why Measure Your Blood Pressure?	
<b>Blood Pressure Standard</b> .....	<b>5</b>
<b>How This Blood Pressure Monitor Works</b> .....	<b>7</b>

## USAGE INFORMATION:

<b>Name/Function of Each Part</b> .....	<b>8</b>
<b>Display Explanations</b> .....	<b>9</b>
<b>Installing Batteries</b> .....	<b>11</b>
<b>Date &amp; Time Set Procedure</b> .....	<b>13</b>
<b>Using The Arm Cuff</b> .....	<b>14</b>
<b>Measurement Procedure</b> .....	<b>18</b>
<b>Risk Category Indicator</b> .....	<b>21</b>
<b>Recalling Values from Memory</b> .....	<b>22</b>
<b>Clearing Values from Memory</b> .....	<b>23</b>
<b>Important Notes Regarding Your Blood Pressure Measurement</b> .....	<b>24</b>
<b>Recalibration Service</b> .....	<b>25</b>

<b>Care, Maintenance &amp; Cleaning</b> .....	<b>26</b>
<b>Potential for Electromagnetic Interference</b> ..	<b>28</b>
<b>Troubleshooting</b> .....	<b>29</b>
<b>Specifications</b> .....	<b>30</b>
<b>Lifetime Limited Warranty</b> .....	<b>31</b>

## IMPORTANT PRODUCT NOTICES AND SAFETY INSTRUCTIONS

When using your blood pressure monitor, basic precautions should always be followed. Please read and follow all instructions and warnings before using this product. Save these instructions for future reference.

- Please note that this is a home healthcare product only and it is not intended to serve as a substitute for the advice of a physician or medical professional.
- Do not use this device for diagnosis or treatment of any health problem or disease. Measurement results are for reference only. Consult a healthcare professional for interpretation of pressure measurements. Contact your physician if you have or suspect any medical problem. Do not change your medications without the advice of your physician or healthcare professional.
- Proper cuff size is critical for accurate measurements. Follow the instructions in this manual and printed on the arm cuff to ensure the appropriate size of cuff is being used.
- This product is not suitable for people with arrhythmias. This device may have difficulty determining the proper blood pressure for individuals with diabetes, poor

circulation of blood, kidney problems or for users who have suffered from a stroke.

- The pulse display is not suitable for checking the frequency of heart pacemakers.
- Electromagnetic interference: The device contains sensitive electronic components. Avoid strong electrical or electromagnetic fields in the direct vicinity of the device (e.g. mobile telephones, microwave ovens). These may lead to temporary impairment of measurement accuracy.
- Use blood pressure monitor only for its intended use.
- Do not wrap the cuff around body parts other than your arm.
- This unit is intended for use in measuring the blood pressure and pulse rate of adults. Do not use this device on infants or those who cannot express their own consent. This unit should not be operated by children.
- Use only 1.5V AA alkaline batteries for power supply.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultatory method, within the limits prescribed by the American National Standard, *Manual, electronic, or automated sphygmomanometers*.

## ABOUT BLOOD PRESSURE

### ***What is Blood Pressure?***

Blood pressure is the pressure exerted on the artery walls while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

### ***Why Measure Your Blood Pressure?***

Among today's various health problems, those associated with high blood pressure are very common. High blood pressure dangerously correlates with cardiovascular diseases. Therefore, blood pressure monitoring is important for identifying those at risk.

## BLOOD PRESSURE STANDARD

The **U.S. Department of Health and Human Services and the National Institutes of Health** have developed a blood pressure standard that identifies areas of low- and high-risk blood pressure. This standard, however, is a general guideline as an individual's blood pressure varies among different people, age groups, etc.

## ***Categories for Blood Pressure Levels in Adults***

	<b>Systolic (mmHg)</b>	<b>Diastolic (mmHg)</b>
Stage 2 Hypertension	≥ 160	≥ 100
Stage 1 Hypertension	140~159	90~99
Prehypertension	120~139	80~89
Normal	< 120	< 80

For adults 18 and older who are not on medicine for high blood pressure, are not having a short-term serious illness, and do not have other conditions, such as diabetes and kidney disease.

To determine category of risk when systolic and diastolic readings fall into two areas, use the higher of the two numbers for classification.

There is an exception to the above definition of high blood pressure for people with diabetes and chronic kidney disease. A blood pressure of 130/80 mmHg or higher is considered high blood pressure for those individuals.

This monitor comes equipped with a Risk Category Indicator to visually indicate the assumed risk level of each reading based upon these standards. See page 21 for more information on this feature.

It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

For reliable monitoring and reference of blood pressure, keeping long-term records is recommended.

## HOW THIS BLOOD PRESSURE MONITOR WORKS


This manual inflate digital blood pressure monitor provides systolic pressure, diastolic pressure and pulse readings quickly and easily. Once the cuff is inflated, the monitor will determine your blood pressures by analysis of your pressure oscillations. When the measurement is completed, the device will deflate the cuff and display your systolic pressure, diastolic pressure and pulse readings.

Based upon your measurement results, the monitor's Risk Category Indicator will alert you if your reading falls into the pre-hypertension, stage 1 hypertension, or stage 2 hypertension risk categories. See page 21 for more information on the Risk Category Indicator.

## NAME/FUNCTION OF EACH PART

## DISPLAY EXPLANATIONS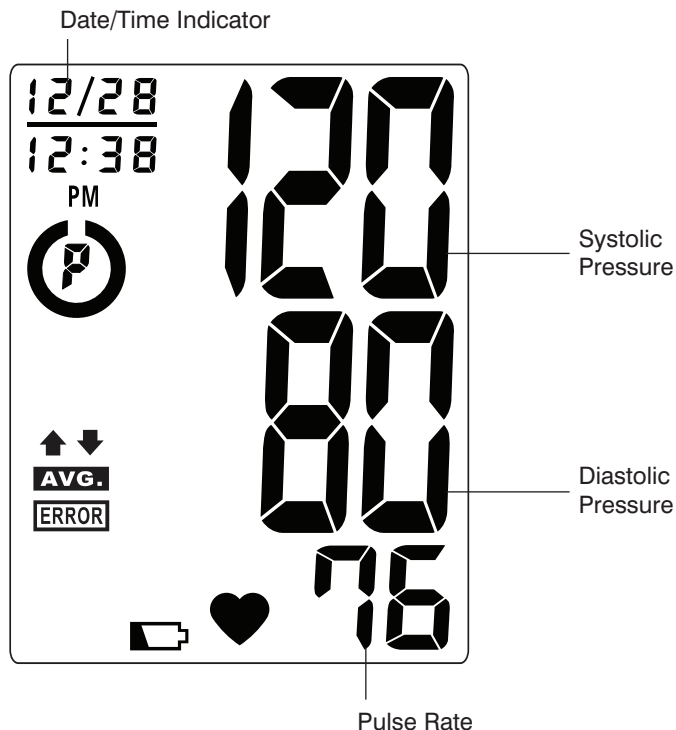

## Display Symbols:


	<b>Inflation Symbol:</b> Appears when cuff requires inflation.
	<b>Deflation Symbol:</b> Appears when cuff is deflating.
	<b>Weak Battery Symbol:</b> Appears when batteries should be replaced.
	<b>Pulse Symbol:</b> Shows the pulse rate per minute.
	<b>Memory Average:</b> Displays average of last 3 readings.
	<b>Risk Category Indicator:</b> Compares readings against blood pressure guidelines. See page 21 for more information.

If **ERROR** and any of the following letters and numbers appear in the area that systolic pressure should be displayed, an error has occurred with your reading. See Troubleshooting section of this manual for more information.

<b>EE</b>	<b>Measurement Error:</b> Measure again. Wrap the cuff correctly and keep arm steady during measurement.
<b>E1</b>	<b>Air Circuit Abnormality:</b> Check cuff connection. Measure again.
<b>E2</b>	<b>Pressure Exceeding 300 mmHg:</b> Turn the unit off to clear, then measure again.
<b>E3</b>	<b>Error Determining Measurement Data:</b> Measure again.
<b>CA</b>	<b>Recalibration Alert:</b> Recalibration recommended. See page 25 for information.

## INSTALLING BATTERIES

1. Press down on latch and lift the battery cover towards you to open the battery compartment.
2. Install or replace 2 “AA” sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery door.

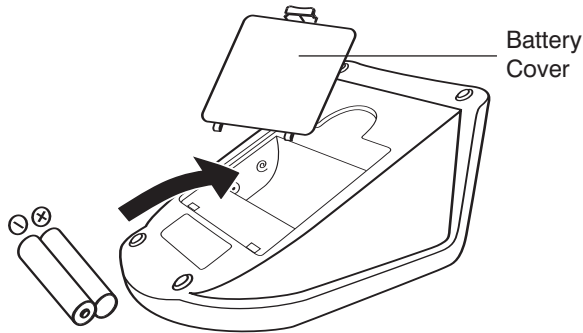## Replace the batteries if:


1. The weak battery symbol appears on the display.
2. Nothing appears on the display when the power is switched on.


As the supplied batteries are for test only, they may be discharged earlier than batteries you buy in stores. Replace the batteries in pairs. Use only 1.5V AA alkaline batteries. Remove batteries when unit is not in use for extended periods of time.


**Note:** Batteries are hazardous waste. Do not dispose of them together with the household garbage.

## DATE & TIME SET PROCEDURE

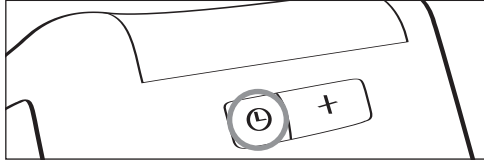1. To adjust the date/time, press the **Set button**  , located on the back of the monitor.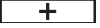

2. The display will show a blinking number showing the date. Change the month by pressing the  **button**. Each press will increase the number by one in a cycling manner. Press the **Set button**  again to confirm the entry, and the screen will show a blinking number representing the day of the month.
3. Change the date and time as described in Step 2 above, using the  **button** to change the numbers and the **Set button**  to confirm the entries.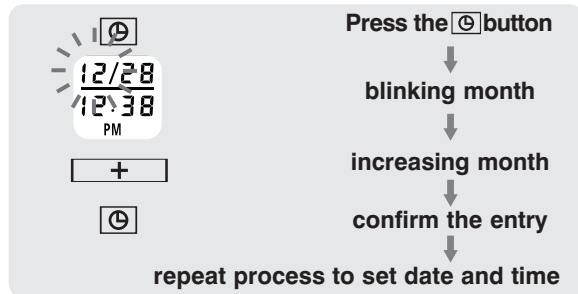


## USING THE ARM CUFF

**Very Important: Proper cuff size is critical to accurate measurement.**

This monitor comes with a **Standard Size** arm cuff that fits arms with circumferences of **9"–13" (23 cm – 33 cm)**.


This cuff is suitable for your use if the colored arrow falls within the solid color line as shown below.

**If the arrow falls outside of the solid color line, you will need a cuff with another circumference. Contact HoMedics Consumer Relations at 1-800-466-3342.**

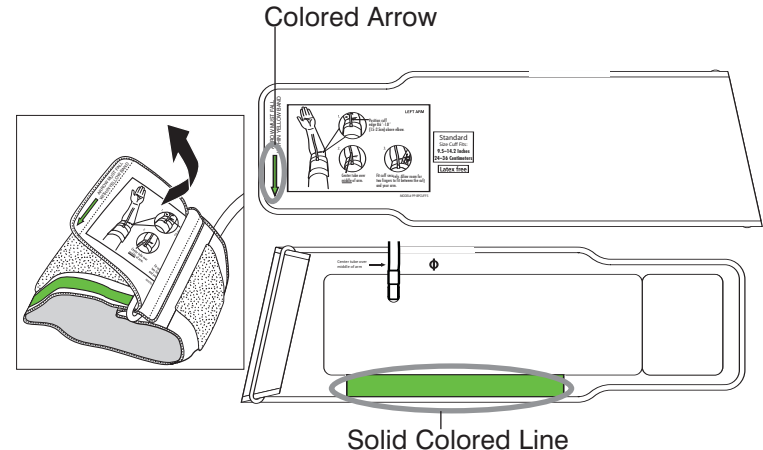

1. If the cuff is not assembled, pass the end of the cuff furthest away from the tubing through the metal D-ring in order to form a loop. The smooth side without the felt material should be on the inside of the cuff loop.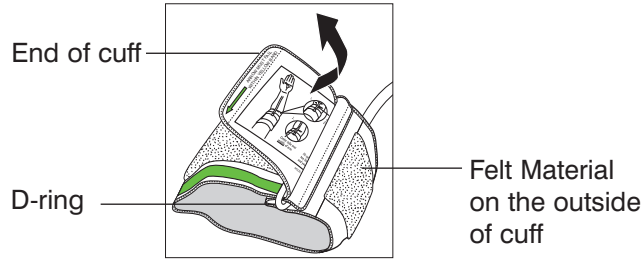


2. Plug the cuff tube into the left side of the unit.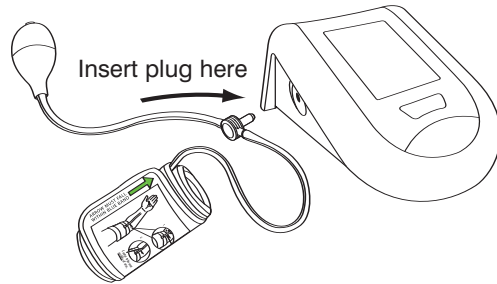


3. Open the arm cuff so that your arm may be placed through it.
4. Remove tight fitting clothing from around your upper left arm.
5. Position cuff edge 0.8-1.2 inches (2-3 cm) above elbow.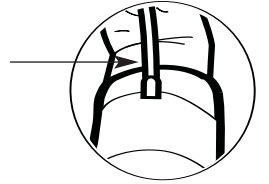


6. Center tube over middle of arm.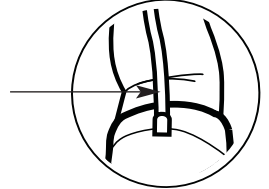

7. Pull the end of the cuff so that it tightens evenly around your arm. Press the hook and loop material together to secure. Allow room for 2 fingers to fit between the cuff and your arm.

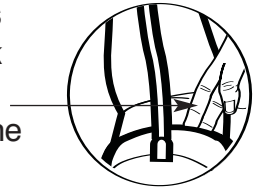
8. Lay your arm on a table (palm upward) so the cuff is at the same height as your heart. Make sure the tube is not kinked.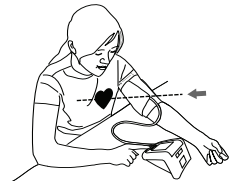

**Note:**


If it is not possible to fit the cuff to your left arm, it can also be placed on your right arm. However, all measurements should be made using the same arm. To use on the right arm, you must position the artery symbol “Φ” over the main artery. Locate the main artery by pressing with two fingers approximately 1" (2 cm) above the bend of your elbow on the inside of your right arm. Identify where the pulse can be felt the strongest. This is your main artery.

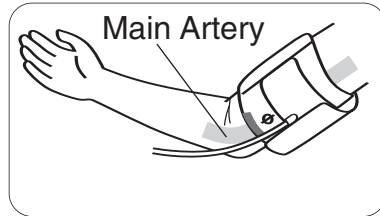**MEASUREMENT PROCEDURE****Important Notes:**

- Blood pressure changes with every heartbeat and is in constant fluctuation throughout the day.
- Blood pressure measurement can be affected by the position of the user, his or her physiologic condition and other factors. For greatest accuracy, wait 1 hour after exercising, bathing, eating, drinking beverages with alcohol or caffeine, or smoking to measure blood pressure.
- Before measurement, it is suggested that you sit quietly for 15 minutes as measurements taken during a relaxed state will have greater accuracy. You should not be physically tired or exhausted while taking a measurement.
- During the measurement, do not talk or move your arm or hand muscles.
- See page 24 for additional notes regarding your blood pressure measurement.

If you are using this blood pressure monitor for the first time, please remove the protective film from the screen.

1. With the cuff wrapped around your arm, press the **START/STOP button**. Do not inflate the cuff unless it is wrapped around your arm. All digits will light up to check the display functions. The checking procedure will be completed after about 3 seconds. The (↓) arrow will appear momentarily and then the display will show a “00”. The monitor is “**Ready to Measure**” at this point.

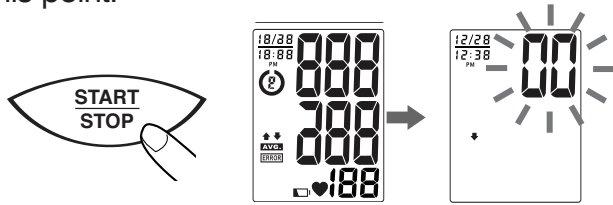2. Using the hand of the arm that does not have the cuff on it (this will be your right hand if wearing the cuff on your left arm), squeeze the inflation bulb and continue pumping until the pressure reading on the display reaches 140 mmHg. The cuff will then begin slowly deflating as part of the measurement process.

If the pressure in the cuff is not enough, an upward arrow (↑) will appear, directing you to continue pumping. It is recommended that you pump until the downward arrow (↓) appears.

3. When the measurement is completed, the cuff will deflate entirely. Systolic pressure, diastolic pressure and pulse will be shown simultaneously on the LCD screen. **The measurement is then automatically stored into memory.**

**Note:**

1. This monitor automatically turns off approximately **1 minute** after last operation. You may also press the **START/STOP button** to turn the unit off.
2. To interrupt the measurement, you may press the **START/STOP button** (recommended), “**MEM**”, or **Date/Time Set** Ⓢ **buttons**. The cuff will deflate immediately after a button is pressed.



## RISK CATEGORY INDICATOR

The U.S. Department of Health and Human Services and the National Institutes of Health have developed a blood pressure standard classifying blood pressure ranges into 4 stages. This unit is equipped with an innovative blood pressure classification indicator, which visually indicates the assumed risk level (normal, prehypertension, stage 1 hypertension, or stage 2 hypertension) of each reading. Symbols are described below and classifications follow the guidelines as described earlier in this manual (page 5).

	Stage 2 Hypertension		Pre-hypertension
	Stage 1 Hypertension		Normal (No Symbol)


## RECALLING VALUES FROM MEMORY

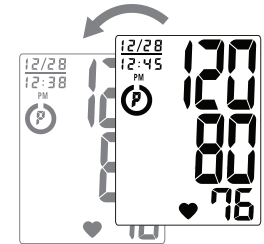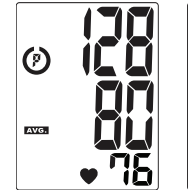1. Press the “MEM” button to access the memory.
2. The monitor will first display the calculated average applied to the last 3 memories.
3. Every new press of the “MEM” button will recall a previous reading. The latest reading will be recalled first.

## CLEARING VALUES FROM MEMORY

Press and hold the **date & time set buttons** (   ) at the same time while in memory mode and the data will be erased automatically.

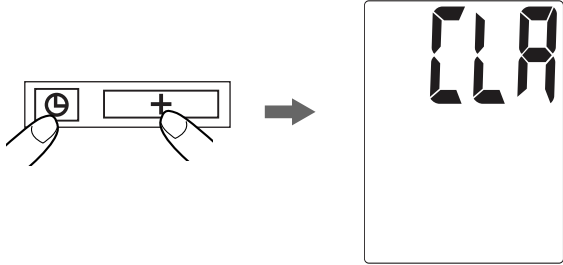## IMPORTANT NOTES REGARDING YOUR BLOOD PRESSURE MEASUREMENT

- Take your reading in a comfortable environment as measurements can be affected by hot or cold temperatures. Take your blood pressure at normal body temperature.
- Do not move or talk during measurement as this can elevate readings.
- Do not move or cross legs during measurement. Keep feet flat on floor.
- Do not touch cuff or monitor during measurement procedure.
- It is suggested that you take your measurements at the same time each day and use the same arm for consistency.
- Users should wait a minimum of 5 minutes before taking additional measurements. More time may be necessary depending upon your physiology.
- **The measurement results that users receive are for reference only. If users have any blood pressure concerns, please consult a physician.**
- **Once inflation reaches 300 mmHg, the unit will deflate automatically for safety reasons.**
- **This product is not suitable for people with arrhythmias.**
- **This device may have difficulty determining the proper blood pressure for users with diabetes, poor circulation of blood, kidney problems, or for users who have suffered a stroke.**

## RECALIBRATION SERVICE

**All digital blood pressure monitors require recalibration in order to ensure continued accuracy.**

This monitor does not require recalibration for 2 years under normal usage conditions (approximately 3 times per day), at which time your monitor will display “CR”. The unit should also be recalibrated if the monitor sustains damage due to blunt force (such as dropping) or exposure to fluids or extreme hot or cold temperature changes.

Please fill in date of first use \_\_\_\_\_  
(*Recalibration is recommended after 2 years.*)

To send monitor for recalibration, please ship to the address below with check or money order in the amount of \$10 payable to HoMedics, Inc. for recalibration service and shipping & handling:

HoMedics Recalibration Service Center  
P.O. Box 7050  
Walled Lake, MI 48390

Allow up to 3 weeks for return of product. For additional information, contact HoMedics Consumer relations at 1-800-466-3342.

## CARE, MAINTENANCE & CLEANING

1. Do not disconnect the tubing or remove the inflation bulb from the arm cuff.
2. Clean the blood pressure monitor body and cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
3. Leaky batteries can damage the unit. Remove the batteries when the unit will not be used for a long time.
4. If the unit is stored near freezing, allow it to acclimate to room temperature before use.
5. This blood pressure monitor is not field serviceable. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems with this device, please contact HoMedics Consumer Relations (contact information can be found on the warranty page).
6. Do not immerse the unit in water as this will result in damage to the unit.

7. Do not subject the monitor and cuff to extreme temperatures, humidity, moisture, and direct sunlight. Protect from dust.
8. Do not fold the cuff and tubing tightly.
9. Do not disassemble the monitor or cuff. If in need of repair, refer to the warranty section of this manual.
10. Do not subject the monitor to extreme shocks (do not drop on floor).
11. Do not inflate the cuff unless wrapped around arm.
12. Do not wrap the cuff around body parts other than your arm.
13. Do not drop or insert any object into any opening or hose.
14. This monitor may not meet its performance specifications if stored or used outside of these temperature and humidity ranges:
  - **Storage Environment**  
Temperature:  
-4°F~140°F (-20°C~60°C)  
Humidity:  
Less than 85% R.H.
  - **Operating Environment**  
Temperature:  
50°F~104°F (10°C ~40°C)  
Humidity:  
Less than 85% R.H.

## POTENTIAL FOR ELECTROMAGNETIC INTERFERENCE

To avoid inaccurate results caused by electromagnetic interference between electrical and electronic equipments, do not use the device near a cell phone or microwave oven.

## TROUBLESHOOTING

If any abnormality arises during use, please check the following points

SYMPTOMS	POSSIBLE CAUSES	CORRECTION
Unit does not turn on when the <b>START/STOP button</b> is pushed.	Batteries have run down.	Replace them with two new AA alkaline batteries.
	Battery polarities have been positioned incorrectly.	Re-insert the batteries in the correct positions.
EE measurement error symbol shown on display or the blood pressure value is displayed excessively low (or high).	Cuff has been placed incorrectly.	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement? Shaking of the arm with the cuff on.	Measure again. Keep arm steady during measurement.
E1 error symbol shown on display	Air circuit abnormality. Cuff tube may not be plugged into monitor correctly.	Check cuff connection. Measure again.
E2 error symbol shown on display	Inflation pressure exceeding 300 mmHg.	Turn the unit off, then measure again.
E3 error symbol shown on display	Error determining measurement data.	Measure again.
CA appears on display	Recalibration alert.	Refer to page 25 for more information.

**Note:** If the unit still does not work, contact HoMedics Consumer Relations. Under no circumstance should you disassemble or attempt to repair the unit by yourself. Contact information for HoMedics Consumer Relations Department can be found on the warranty page.

## SPECIFICATIONS

<b>Power Source:</b>	Two 1.5V DC (AA) batteries
<b>Measurement Method:</b>	Oscillometric
<b>Measurement Range:</b>	Pressure: 40~280 millimeters mercury (mmHg) Pulse: 40~199 beats/minute
<b>Accuracy:</b>	Pressure: $\pm 3$ mmHg Pulse: $\pm 5\%$ of reading
<b>Pressure Sensor:</b>	Semi-conductor
<b>Inflation:</b>	Manual Inflation
<b>Deflation:</b>	Automatic Pressure release valve
<b>Memory Capacity:</b>	99 memories
<b>Auto-shut-off:</b>	1 minute after last button operation
<b>Operation Environment:</b>	Temperature: 50°F~104°F (10°C ~40°C) Humidity: Less than 85% R.H.
<b>Storage Environment:</b>	Temperature: -4°F~140°F (-20°C~60°C) Humidity: Less than 85% R.H.
<b>Weight:</b>	.35 lbs/159 grams (without batteries)
<b>Arm Circumference:</b>	Standard size adult arm cuff: 9"-13" (23 – 33 cm)
<b>Dimensions:</b>	5.3"(L) x 3.5"(W) x 2.3"(H) 135mm(L) x 90mm(W) x 58mm(H)
	Type BF Device and cuff are designed to provide special protection against electrical shocks.

**Note:** These specifications are subject to change without notice.

**For service or repair,  
do not return this unit  
to the retailer. Contact  
HoMedics Consumer  
Relations at:**

**Mail:  
HoMedics, Inc.  
Service Center  
Dept. 168  
3000 Pontiac Trail  
Commerce Township,  
MI 48390**

**Email:  
cservice@  
homedics.com**

**Phone:  
1-800-466-3342  
Business Hours:  
8:30am-5pm ET  
Monday-Friday**

**Distributed By HoMedics  
for Walgreen Co.,  
Deerfield, IL  
60015-4616**

**HoMedics, Inc.  
3000 Pontiac Trail,  
Commerce Township, MI  
48390**

Printed in China

LIMITED LIFETIME WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for the life of the original purchaser from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations  
Service Center Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI 48390

No COD's will be accepted

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein.

This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

**lifetime  
limited warranty**

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

©2008 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics is a registered trademark of HoMedics, Inc. and its affiliated companies. All rights reserved.  
IB-518732

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