HoMedics Service Center
1.800.466.3342
8:30 – 5:00 p.m. (hora del ESTE) de lunes a viernes

Dirección postal:
HoMedics
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI 48390

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Instruction Manual and Warranty Information

1 year limited warranty

ICUSH-100

HOMEDICS

IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING DANGER- TO REDUCE THE RISK OF ELECTROCUTION:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink. DO NOT place in or drop into water or other liquid.
- NEVER use pins or other metallic fasteners with this appliance.
- Keep dry - DO NOT operate in a wet or moist condition.

WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics.

- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Keep cord away from heated surfaces.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- To disconnect, turn all controls to the "off" position, then remove plug from outlet.
- This appliance is designed for personal, non-professional use only.
- DO NOT use outdoors.
- Temperatures sufficiently high to cause burns may occur regardless of the control setting. DO NOT use on an infant or invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
- DO NOT crush. Avoid sharp folds.

SAVE THESE INSTRUCTIONS

Caution—Please read all instructions carefully before operating.

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- Do not use this product before bed. The massage has a stimulating effect and can delay sleep.
- Never use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.
- This product is not for use in automobiles.

Automatic Shut Off

We've designed this back massager for convenience and safety. As a special precaution against falling asleep while using the unit, or leaving the unit on while it's still in operation, we've programmed it to automatically stop massaging 15 minutes after receiving your last command. To reactivate the massage cushion, simply press any button. Any time you change massage functions, the timer will reset. This special feature should not be considered as a substitute for OFF. Always remember to turn your unit off when not in use.

Instructions for Use

For Use in Home or Office

Plug adapter into a 120 volt AC outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach the massage cushion to most chairs for an invigorating massage as you read, rest or even work.

Strapping System

This back massager incorporates a unique strapping system which allows you to fasten it to most any chair. Pull upper and lower straps around chair and fasten with clips. Your massager won't slip or slide away.

Speakers

Two adjustable aluminum cone speakers play your audio. Speakers are activated when the main power is on and an audio source is connected to the audio input cord.

Maintenance

To Store

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the controller cord.

To Clean

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, paint thinner to clean.
- Do not attempt to repair the massager. There are no user serviceable parts. For service, send the unit to the HoMedics address listed in the warranty section.

Note:

Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using in upholstered furniture.

Caution:



All servicing of this massager must be performed by authorized HoMedics service personnel only.

Note:




You can play speakers with or without massage

Caution:


Do not operate speakers at high volume to prevent possibility of hearing loss.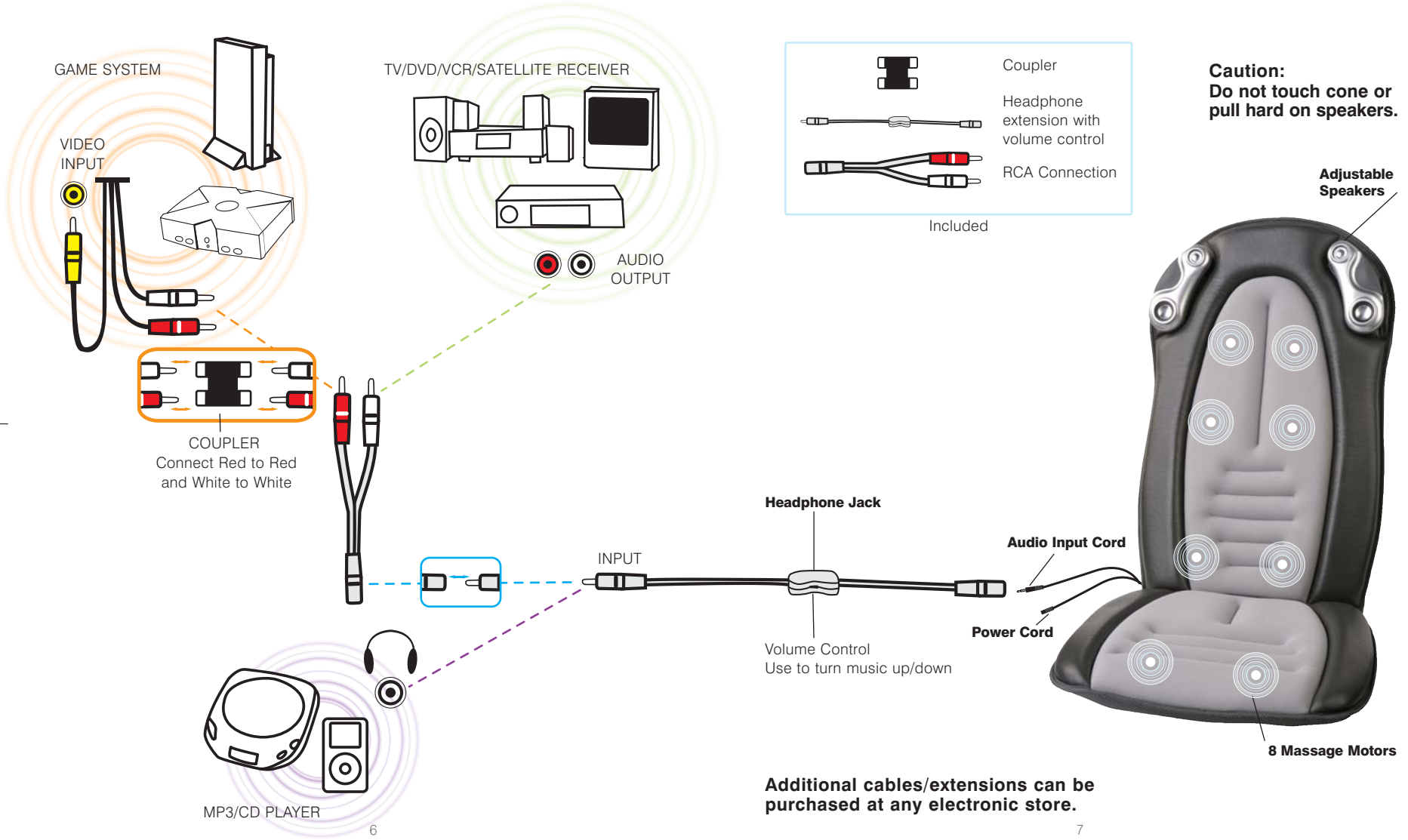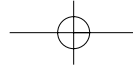

	Coupler
	Headphone extension with volume control
	RCA Connection
Included	

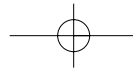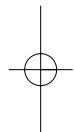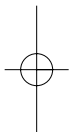Additional cables/extensions can be purchased at any electronic store.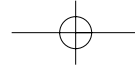

Power Adapters

This product is powered by a custom 120 volt AC home adaptor.

Note: The hole in the top of the control is only for demo purposes.

Power Button

To turn on the audio/message functions, press the POWER button. The L.E.D. indicator above it will illuminate and confirm your selection. To turn the audio/message functions off, simply press the button again and the L.E.D. indicator will turn off.

Audio Sync Function

Audio sync feature synchronizes your massage to the beat of the music or game. The glowing light around the button indicates that music sync is activated. See Message Functions. While in audio sync all 3 programmed massage functions can operate. Audio sync must be reactivated once all 3 programs have been cycled.

Speaker L.E.D.s

Blue L.E.D.s will light up when iCUSH is powered and ready to play audio.

Massage Functions

This product incorporates 3 pre-programmed massage functions that can be easily selected by pressing the corresponding button. Once activated, you can move from one function to another by simply pressing another function key. The fourth press of the button will turn off massage.

All Motor Massage

Activates all 8 motors for a total massage experience. This is the default setting when the unit is powered on.

Up and Down

The top to the bottom, back and forth

Mix Massage

An invigorating mix of the other programs

Targeted Massage Zones





Zone buttons activate motors to massage specific areas: shoulders, upper back, middle back, lower back, and thighs. Customize your massage by choosing one specific area, any combination of areas or all five at once. To select a massage area, simply push the button and the corresponding L.E.D. light will illuminate. To deselect, simply push the button again and the corresponding L.E.D. lights will turn off.

Massage Speed

Choose the speed at which your custom programmed massage travels in the Up and Down and Mix programs. There are 3 options: low (L), medium (M) and high (H). L.E.D. lights will indicate your selection. The default setting is low. To change the speed of your massage simply press the button.

Massage Strength

Choose the intensity of your massage with three options of massage strengths: low (L), medium (M), and high (H). L.E.D. lights correspond to your selection. The default setting is low. To change the strength of your massage, simply press the button again.

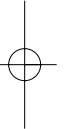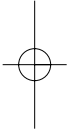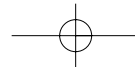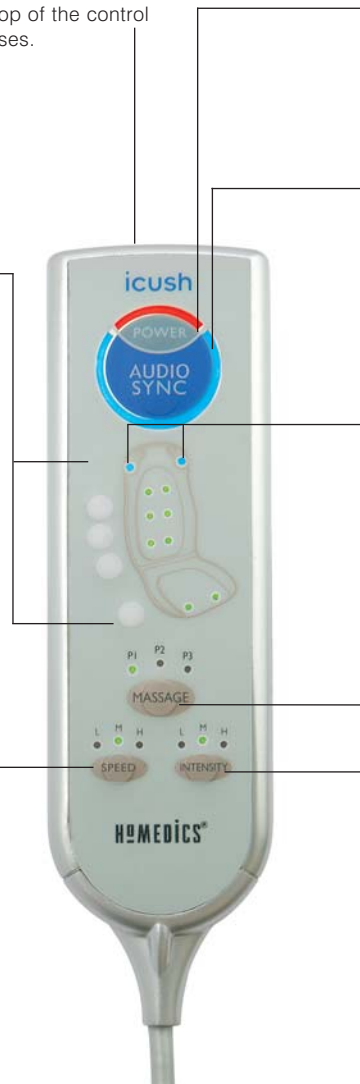LIMITED ONE YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of one year from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers. To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

©2006 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies.
iCush™ is a trademark of HoMedics, Inc. and its affiliated companies. The beauty of living well.™ is a trademark of HoMedics, Inc. iPod™ is a trademark of Apple Computer, Inc. XBOX™ is a trademark of Microsoft Corporation. PlayStation® is a registered trademark of Sony Computer Entertainment Inc.

All rights reserved.
IB-ICUSH100A

HoMedics Service Center
1.800.466.3342
8:30 – 5:00 p.m. (EST) M – F

Mail To:
HoMedics
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

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