

HoMEDICS

TWO YEAR LIMITED WARRANTY

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

Mail To:

Consumer Relations
HoMedics
Service Center, Dept. 168
3000 Pontiac Trail
Commerce Township, MI
48390

e-mail:
cservice@homedics.com

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA,
please visit: www.homedics.com

©2002 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies.
BackRejuvenator™ and The beauty of living well.™ are trademarks of HoMedics, Inc. and its affiliated companies. All rights reserved.
IB-BK900

2 year
limited warranty

HoMEDICS®

The beauty of living well.™

BackRejuvenator™

**8-motor Back Massager
with Percussion Action**



**Instruction Manual and
Warranty Information**

BK-900

2 year
limited warranty

IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING DANGER- TO REDUCE THE RISK OF ELECTROCUTION:

- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.


WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.

- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- NEVER use pins or other metallic fasteners with this appliance.
- Keep cord away from heated surfaces.
- NEVER block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- To disconnect, turn all controls to the "OFF" position, then remove plug from outlet.
- This appliance is designed for household use only.

SAVE THESE INSTRUCTIONS **Caution— Please read all instructions carefully before operating.**

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the products excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.

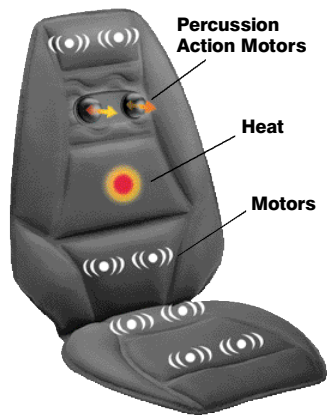

- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children or invalids without adult supervision.

Instructions for Use

For Use in Home or Office

Plug adaptor into a 120 volt AC outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach Back Rejuvenator to most chairs for a luxurious massage as you read, rest or even work.

For Use in Your Car

Simply plug the 12 volt auto adapter into a cigarette outlet and plug the receptacle end into the input jack, located along the side of the cushion. Slip Back Rejuvenator's straps over your seat to enjoy a relaxing massage as you drive.

Caution:
All servicing of this massager must be performed by authorized HoMedics service personnel only.

Strapping System

Back Rejuvenator incorporates a unique strapping system which allows you to fasten it to almost any chair or auto seat. Just slip the straps over the seat or chair and adjust the hook and loop straps to secure. Your massager won't slip or slide away.

Maintenance

To Store Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the power cord around the unit. DO NOT hang the unit by the controller cord.

To Clean Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean.
- Do not attempt to repair Back Rejuvenator massager. There are no user serviceable parts. For service, send the unit to the HoMedics address listed in the warranty section.

Please Note:

For your safety, this product is equipped with a pressure activated switch to control the heat function. **Heat will not operate unless pressure is applied to this switch, located on the seat bottom of the unit.** Once pressure is removed, heat will turn off. After applying pressure, please allow 5-10 minutes for heat to reach maximum temperature.

BackRejuvenator™

8-motor Back Massager with Percussion Action

Power Button

To turn on the massage functions, press the POWER button. The L.E.D. indicator above it will illuminate and confirm your selection. To turn the massage functions off, simply press the button again and the L.E.D. indicator will turn off.

Power Adapters

Back Rejuvenator is powered by a custom 120 volt AC home adaptor, and 12 volt DC auto adaptor (cigarette lighter plug-in style). Both are included with your unit.

NOTE: When selecting the heat feature with massaging action, there will be a slight decrease in the strength of the massage output. This adjustment is perfectly normal and should not be interpreted as a defect.

Targeted Massage Buttons

Zone buttons activate motors to massage specific areas: neck and shoulders, lower back, buttocks and thighs. Customize your massage by choosing one specific area, any combination of areas or all four. To select a massage area, simply push the button and the corresponding L.E.D. light will illuminate. To deselect, simply push the button again and the corresponding L.E.D. lights will turn off.

2 Percussion Action Heads

Push either blue button to activate the dual pivoting heads for an invigorating massage to your upper back. Increase the massage speed by pushing the arrow button on the right. Decrease the speed by pushing the arrow button on the left.



Heat

Soothing, penetrating heat is activated by pushing the heat button. To turn heat off, simply push the button again. A glowing L.E.D. light indicates that heat is on.

NOTE: The percussion action feature and the HEAT function are pressure activated. The massage/heat action will not function unless pressure is applied to the back of the cushion. Once pressure is removed, the percussion action massage or heat will stop.

Massage Speed

Choose the speed at which your custom programmed massage travels up and down Back Rejuvenator. With each press of the button, the cycle speed increases, creating three levels: low (L), medium (M), and high (H). Corresponding L.E.D. lights will indicate your selection. To decrease speed, simply press the button again.

Massage Strength

Choose the intensity of your massage. Back Rejuvenator gives you three options of massage strengths: low (L), medium (M), and high (H). L.E.D. lights correspond to your selection. To change the strength of your massage, simply press the button again.

Massage Functions

Back Rejuvenator incorporates 6 preprogrammed massage functions that can be easily selected by pressing the corresponding button. Once activated, you can move from one function to another by simply pressing another function key. All 6 programs allow you to choose the body zones activated, and the speed and intensity of your massage.

All Motor Massage

Activates all 8 vibration motors for a total massage experience

Cycle Massage

Massages from your neck down to your thighs and back up

Tapping Massage

Repeating bursts of massage relief

Side-to-Side Massage

From your left side to your right, back and forth

Zig Zag Massage

Massage pulses move in a stimulating "Z" pattern

Random Massage

Carries pulses in a random sequence