HOMEDICS

THE LEADERS IN HEALTH & WELLBEING

BODY SHAPING MASSAGER

PERCUSSION MASSAGE AND SOOTHING HEAT
TO HELP SHAPE, FIRM, AND SMOOTH

PLEASE TAKE A
MOMENT NOW

REGISTER YOUR PRODUCT AT:

www.homedics.com/register

YOUR VALUABLE INPUT REGARDING
THIS PRODUCT WILL HELP US CREATE
THE PRODUCTS YOU WILL WANT
IN THE FUTURE.

INSTRUCTION MANUAL AND WARRANTY INFORMATION

ONE YEAR LIMITED WARRANTY

CELL-200

IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING

DANGER – TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug immediately.
- DO NOT use while bathing or in a shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink.
- DO NOT place in or drop into water or other liquid.

WARNING – TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE OR INJURY TO PERSONS:

- This appliance has heated and cold attachments. Persons insensitive to heat or cold must be careful when using this appliance.
- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Not for use by children.
- Close supervision is necessary when this appliance is used by, on or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return this appliance to HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- Never use pins or other metallic fasteners with this appliance
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as handle.
- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- NEVER operate on a soft surface such as a bed or couch where the air openings may be blocked.

- Use heated surfaces carefully. May cause serious burns. DO NOT use over insensitive skin areas or in the presence of poor circulation. The use of heat by children or incapacitated persons may be dangerous.
- Keep long hair away from massager while in use.
- DO NOT use massager in close proximity to loose clothing or jewelry.



SAVE THESE INSTRUCTIONS

CAUTION—PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.


- This massager is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product, if
 - You are pregnant
 - You have a pacemaker
 - You have any concerns regarding your health
- Not recommended for use by Diabetics.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- NEVER use this product in automobiles.
- This appliance is intended for household use only.

PLEASE NOTE: THE SOOTHING HEAT OPTION WILL ONLY WORK WHILE A MASSAGE FEATURE IS SELECTED.


BODY SHAPING MASSAGER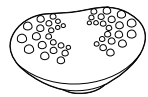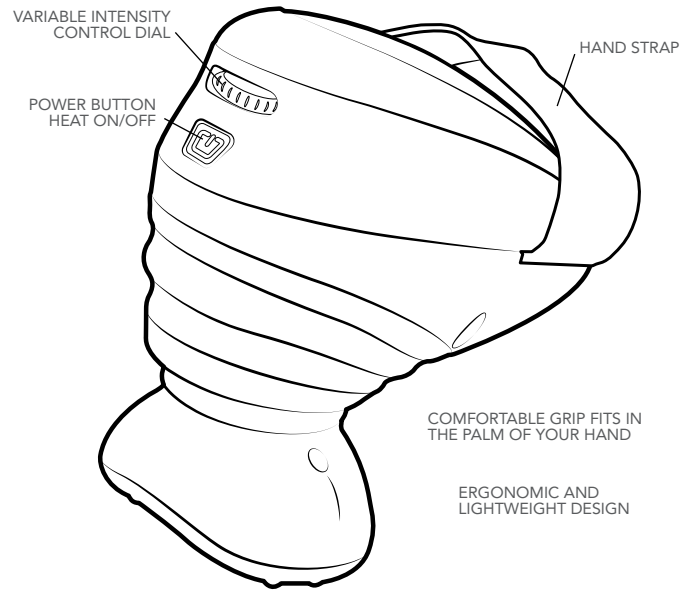


SMALL NODE ATTACHMENT

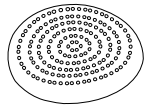LARGE NODE ATTACHMENT

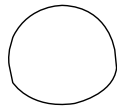DOME ATTACHMENT

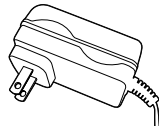AC ADAPTER

UNIQUE FEATURES & SPECIFICATIONS

VARIABLE INTENSITY PERCUSSION MASSAGE

Shapes, firms and smooths different areas of your body

OPTIONAL HEAT

Soothes and boosts effectiveness of targeted massage

3 ATTACHMENTS

Designed to target different areas of the body:

1. Dome – Spot massage relief
2. Small Node – Massage of smaller areas, ie, arms
3. Large Node – Massage of larger areas, ie, legs

HAND STRAP


Fits to the palm of your hand for easy operation

AC ADAPTER INCLUDED

HOW TO USE


BEFORE TURNING THE UNIT ON

- Make sure that the unit is set to the OFF position before plugging into the outlet.
- Select the desired attachment and install it on the unit prior to use.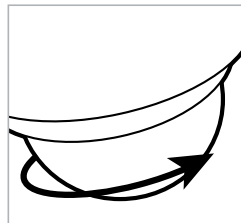

CHANGING THE ATTACHMENTS

1. Be sure the unit is OFF and unplugged prior to changing the attachment. Remove the attachment head from the massager by twisting it counter-clockwise and set aside.

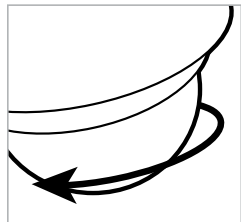2. Attach the desired attachment head to the massager by twisting it clockwise until secure.

TURNING ON THE MASSAGER

1. Plug unit into a household AC electrical outlet. Slide your hand into the elastic hand strap with fingers outstretched until the massager rests comfortably in the palm of your hand.


HOW TO USE

2. Press the POWER button once to turn the massager ON. The button will illuminate BLUE

3. Adjust the massage intensity by rotating the dial to the right for higher intensity, and to the left for a lighter massage.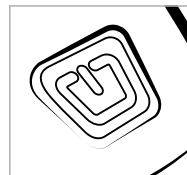

4. To activate HEAT, press the POWER button again. It will illuminate RED to indicate that the heat setting is on. The center of the attachment will begin to heat up.

NOTE: To eliminate the risk of being burned, never use the heat function without the attachments. Do not use the heat function continuously for more than 15 minutes. Do not use the heat function if you have sensitive skin, poor circulation, discolored areas, swelling, burning and inflammation, or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.

5. Touch the massager to the area you wish to treat. Move the unit slowly, holding it over the targeted area. Massage in a firm circular motion, concentrating on targeted problem areas of the body. Treat the area for a short period of time and move to another area. DO NOT treat the same area for longer than 3 minutes.

To turn the unit OFF, press the POWER button a third time. For your safety the device will automatically turn off after 15 minutes of continuous use.

CLEANING AND CARE

CAUTION: before cleaning the unit, turn power off and unplug, allowing the unit to cool before cleaning.

TO CLEAN THE SURFACE

Clean the surface of the unit with a clean, damp, soft cloth.

NOTE: Never allow water or any other liquids to come in contact with the unit.

- Do not immerse in any liquid to clean
- Never use abrasive cleaners to clean the unit.

TO STORE

Place components into its box and store in a safe, dry place. DO NOT wrap the power cord around the unit. DO NOT hang the unit by the cord.

TROUBLESHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
Unit will not operate	<ul style="list-style-type: none">• Unit is not plugged in• No power to unit	<ul style="list-style-type: none">• Plug unit in• Press the ON/OFF button to turn the power on• Check circuits, fuses, try a different outlet.

FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This class B digital device complies with Canadian ICES-003.

ONE YEAR LIMITED WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of one year from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, contact a consumer relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you. For more information regarding our product line in the USA, please visit: www.homedics.com. For Canada, please visit: www.homedics.ca.

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