

**THE
SHARPER
IMAGE®**

**Personal
Percussion
Massager**
with Heat & Gel Nodes

HF768 Instructions

Read and save these instructions.







Dear Sharper Image Customer,

Thank you for your purchase of a Sharper Image product. We appreciate your business very much.

Our goal is that you will be completely satisfied with this product's quality and performance. We offer generous return privileges and excellent warranties on all our products.

Since 1977, The Sharper Image has shared the fun of discovering innovative products that make life better and more enjoyable. We've been in business a long time because we are an exciting place to shop and — just as importantly — we strive to deliver outstanding customer service that exceeds your expectations.

Enjoy your purchase!

Call 1-800-344-4444 for ordering and customer service, or visit
www.sharperimage.com, *our award-winning Website.*





Table of Contents

Important Safeguards	6
Caution.....	9
General Instructions.....	11
Parts Diagram.....	11
Operation	12
Changing Attachments	14
Maintenance.....	15

IMPORTANT SAFEGUARDS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE OBSERVED, INCLUDING THE FOLLOWING:

DANGER — TO REDUCE THE RISK OF ELECTROCUTION:

- Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Do not use while bathing or showering.
- Do not place or store appliance where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Never use pins or other metallic fasteners with this appliance.
- Keep dry. Do not operate in wet or moist conditions.

IMPORTANT SAFEGUARDS

WARNING — TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from the outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual.
- Do not use attachments not recommended by the manufacturer; specifically any attachments not provided with the unit.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to a qualified service center for examination and repair.
- Keep cord away from heated surfaces.
- Never use while sleeping or fall asleep while using the massager.

IMPORTANT SAFEGUARDS

- Never drop or insert any object into any opening.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- Do not carry this appliance by supply cord or use cord as a handle.
- Do not use the percussion action massager directly on one concentrated area of the body. Percussion action massagers are designed to be used in a continuous motion, constantly changing the massage area.
- To disconnect, turn all controls to the **OFF** position, then remove plug from outlet.
- This appliance is designed for household use only.
- Do not use outdoors.
- Do not use on an infant or invalid, or on a sleeping or unconscious person.



IMPORTANT SAFEGUARDS

- Do not use on sensitive skin or on a person with poor blood circulation.
- Connect this appliance to a properly grounded outlet only. See grounding instructions.

Caution

Please read all instructions carefully before operating:

- If you have any concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.

Caution


- Never use this appliance on or near eyes or other highly sensitive areas.
- Never use this product directly on swollen or inflamed areas, or on skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- Do not use this product before bed. The massage has a stimulating effect and can delay sleep.
- Never use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls, or by anyone who has sensory deficiencies in their body.
- This unit should not be used by children or invalids without adult supervision.
- All servicing of this unit must be performed by authorized service personnel only.

Caution

SAVE THESE OPERATING INSTRUCTIONS

General Instructions

Parts Diagram

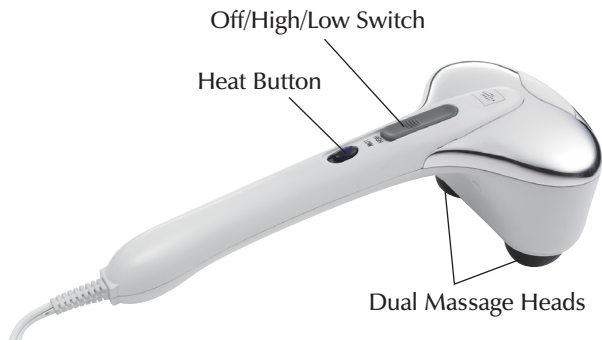

Operation

Caution: Never use the heat feature without the proper attachment. To reduce the risk of electric shock, this appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. This unit is powered by and comes equipped with a UL approved cord. Always make sure the unit is set to the OFF position before inserting the plug into a 120V electrical outlet.
2. **Massage:** To turn the unit on, slide the black **ON/OFF** switch, located along the top of the unit's handle, up. Slide the switch up one position for the low-intensity setting and up two positions for the high-intensity setting. To turn the unit off, slide the switch to the **OFF** position.
3. **Massage with Thermal Heat:** Locate the two red (heat) attachments. Attach to the massage heads by turning clockwise to secure. Slide the red **HEAT** button up for heat. Allow a few minutes for heat to fully activate. To turn the heat off, slide the switch to the **OFF** position.

Operation

Note: The heat option only works while a massage feature is selected. To eliminate the risk of being burned, never use the heat function without the attachments. Do not use the thermal heat continuously for more than 15 minutes. Do not use the thermal heat if you have sensitive skin, poor circulation, discolored areas, swelling, burns, inflammation or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.

4. Apply massager head lightly to the area you wish to treat.
5. Move the unit slowly, remaining in the targeted area. Treat the area for a short period of time and move to another area. Never treat the same spot longer than 3 minutes.

Caution: As stated in the Important Safeguards section of this manual, never use this appliance on or near eyes or other highly sensitive areas.

6. Operation of longer than 15 minutes is not recommended and may cause overheating.
7. When treatment is complete, turn the unit to the **OFF** position, allow to cool, unplug the power supply and store properly in a dry place.

Changing Attachments

Your massager comes equipped with five custom massage head attachments. Each set is a different firmness:

Black – Soft

Dark Gray – Medium hardness

Light Gray – Firm

Red – Heat

Blue – Gel

To change to a different set, simply twist off (counterclockwise) the set that is attached to the unit and twist on (clockwise) the desired set.




Maintenance

To Clean

Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. Never allow water or any liquids to come into contact with the handle. Periodically straighten the cord if it becomes twisted.

To Store

Unplug the appliance from the outlet and allow it to cool before storing in its box or in a clean, dry place. Coil the cord and secure it with the included fastener. Do not hang the unit by the power cord.





Warranty

What Is Covered

This warranty covers any defects in materials or workmanship, with the exceptions stated below.

How Long Coverage Lasts

This warranty runs for 90 days from the date of original purchase.

What Is Not Covered

This warranty does not cover damage caused by misuse or use other than as intended and described in the product instruction manual, or loss or damage to batteries or removable parts.

What The Sharper Image Will Do

The Sharper Image warrants this product against defects in materials and workmanship. When repair is not practical, The Sharper Image will have the option to replace the defective product with one that is the same or has similar functionality.

How to Obtain Service

Return the product and receipt, along with a brief explanation of the problem, to:

T.S.I. - Returns Department
2901 West 60th Street
Little Rock, AR 72209

If you have questions, please feel free to contact our customer service representatives at **1-800-344-5555** or by email at **care@web.sharperimage.com**.

This warranty gives you specific legal rights, and you may have other legal rights that vary from state to state.

© 2007 Sharper Image Corporation

Sharper Image Corporation
350 The Embarcadero, Sixth Floor
San Francisco, CA 94105-1218

Shop online: **www.sharperimage.com**

Actual product may differ slightly from the product described herein.

V2 03/07