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**HoMedics**  
**Service Center, Dept. 168**  
**3000 Pontiac Trail**  
**Commerce Township, MI**  
**48390**

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# Massaging **Slippers**

With Smart Foam™

**Instruction Manual and  
Warranty Information**

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**1 year  
limited warranty**

## IMPORTANT SAFEGUARDS:

**When using an electrical appliance, basic precautions should always be followed, including the following:**

### READ ALL INSTRUCTIONS BEFORE USING

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- This unit should not be used by children without adult supervision.
- Turn off appliance when not in use.
- Never operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics Service Center for examination and repair.
- Keep away from heated surfaces.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- This appliance is designed for household use only.

## SAVE THESE INSTRUCTIONS

**Caution: Please read all instructions carefully before operating.**

- If you have concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.

**Caution: All servicing of this massager must be performed by authorized HoMedics Service Personnel only.**

- Those with the following conditions should consult their doctor before using this product:
  - Pregnancy
  - Diabetes
  - Heart Disease
  - Malignant Tumor
  - Conditions requiring rest
- If at any time soreness, rash or an increase in pain occurs, discontinue use and consult your physician.
- NEVER use this product directly on swollen or inflamed areas, skin eruptions or an open wound.
- A massage session longer than 15 minutes is not recommended.

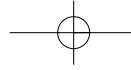
## Battery Precautions

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of batteries may cause damage to the unit.
- Do not mix different types of batteries together (e.g., alkaline with carbon-zinc or old batteries with new ones).
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.

## Battery Installation

1. Roll back fabric lip along back side of slippers and open the zipper to expose the battery compartment. Remove battery cover.
2. Install two "AA" batteries (not included), observing +/- polarities diagram in the battery compartment.
3. Replace battery cover and close zipper.

Note: Two "AA" batteries must be installed in each slipper.

## Instructions for Use

1. Place your feet inside each slipper.
2. Massage may be activated by pressing the On/Off button on the inside of each slipper (see pg. 5).
3. To turn off the massage, simply depress the On/Off button once again.

Note: It is not recommended that the massage function be activated while you are walking in the slippers. It is best enjoyed while you are sitting and relaxing. You may use the slippers with or without massage.

## Maintenance

### TO CLEAN:

Clean only with a soft, slightly damp cloth. Never allow water or any other liquids to come into contact with the unit.

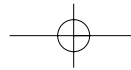
- Do not immerse in any liquid to clean. To clean the fabric cover, wipe with damp sponge, using a mild liquid detergent or suds. Wipe surface with a clean, dry cloth.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean the massager.
- Do not attempt to repair this unit. There are no user serviceable parts. For service, send the unit to the HoMedics address listed in the warranty section.

### TO STORE:

Place massager in its box or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or snag the fabric surface.

# Massaging Slippers

With Smart Foam™

**ONE YEAR LIMITED WARRANTY**  
(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of one year from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of one year from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

**For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)**

© 2005 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies. The beauty of living well.™ and Smart Foam™ are trademarks of HoMedics Inc. and its affiliated companies. All rights reserved.  
IB-MS1-A

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MS -1

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**Mail To:**

**HoMedics  
Service Center, Dept. 168  
3000 Pontiac Trail  
Commerce Township, MI  
48390**

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