HOMEDICS

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**Instruction Manual and
Warranty Information**

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**1 year
limited warranty**

HoMEDICS

**IMPORTANT SAFEGUARDS
WHEN USING ELECTRICAL PRODUCTS,
ESPECIALLY WHEN CHILDREN ARE PRE-
SENT, BASIC SAFETY PRECAUTIONS
SHOULD ALWAYS BE FOLLOWED,
INCLUDING THE FOLLOWING:**

**Caution:
All servicing of
this massager
must be
performed by
authorized
HoMedics
Service
Personnel only**

**READ ALL INSTRUCTIONS BEFORE
USING**

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual.
- Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- This unit should not be used by children without adult supervision. THIS IS NOT A TOY.
- Turn off appliance when not in use.

- Never operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics Service Center for examination and repair.
- Keep away from heated surfaces.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- This appliance is designed for household use only.

SAVE THESE INSTRUCTIONS

Caution: Please read all instructions carefully before operating.

- If you have concerns regarding your health, consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Those with the following conditions should consult their doctor before using this product:
 - Pregnancy
 - Diabetes
 - Heart Disease
- If at any time soreness, rash or an increase in pain occurs, discontinue use and consult your physician.
- NEVER use this product directly on swollen or inflamed areas, skin eruptions or an open wound.
- A massage session longer than 15 minutes is not recommended.

Battery Precautions

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of battery may cause damage to the unit.
- Do not mix different types of batteries together (e.g., alkaline with carbon-zinc or old batteries with new ones).
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.
- Do not dispose of batteries in fire. Batteries may explode or leak.
- Do not immerse battery compartment in water.

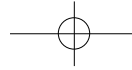
Figure A

Figure B

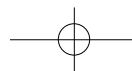
Figure C

NOTE: Water Bottle may be used without massage function.

Operating Instructions:

- Unscrew water bottle cap (A) and remove from water bottle bag. Unscrew water massage cap (B) and place 2 “AA” batteries into the chamber as indicated on compartment. Replace massage cap.
- Fill water bottle two-thirds (60 oz) full with plain tap water only (No more than 128 F).
Note: Do Not overfill.
Note: Do Not use boiling water.
- Replace water bottle cap and make sure it is on securely
- Test for leakage by turning bottle upside down and pressing firmly.
- To activate massage press power (C)
- Hold Water Bottle over desired area to be massaged.
- After completing massage turn Water Bottle off by depressing power button once.
- Empty water bottle when not in use, by unscrewing water bottle cap and removing it from bag. Pour water into the sink.

Removable Cover

- To remove cover, first make sure water bottle is empty.
- Remove water bottle cap / massage tube.
- Pull hook and loop closure apart (Figure 1) and pull water bottle out through this opening. (Figure 2)
- To replace cover, make sure water bottle is empty. Pull hook and loop closure apart and slide water bottle in bottom first (Figure 3). Bend top part and push the top through the top opening. Secure hook and loop closure together.

Figure 1

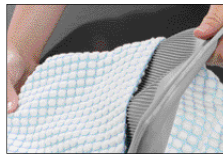
Figure 2

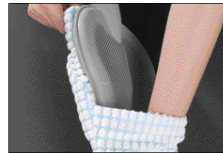
Figure 3

Recommended Cleaning Instructions:

Removable Cover:

- Machine Wash Warm, Tumble Dry Low

Water Bottle

- Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the battery compartment.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass / furniture polish or paint thinner to clean.
- Do not attempt to repair the Water Bottle. There are no user serviceable parts. For service send the unit to the Homedics address listed in the warranty section.

Mail To: HoMedics Service Center, Dept. 168
3000 Pontiac Trail
Commerce Township, MI 48390
e-mail:
cservice@homedics.com
phone: 800-466-3342
hours: Monday – Friday
8:30am to 5:00pm EST

LIMITED ONE YEAR WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of one year from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to Retailers.

To obtain warranty service on your HoMedics product, mail the product and your dated sales receipt (as proof of purchase), postpaid, to the following address:

HoMedics Consumer Relations
Service Center Dept. 168
3000 Pontiac Trail
Commerce Township, MI 48390

No COD's will be accepted.

HoMedics does not authorize anyone, including, but not limited to, Retailers, the subsequent consumer purchaser of the product from a Retailer or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS WHICH ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged and/or resealed products, including but not limited to sale of such products on Internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof which are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights which may vary from country to country. Because of individual country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: www.homedics.com

© 2006 HoMedics, Inc. and its affiliated companies, all rights reserved. HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies. H2Olistic is a trademark of HoMedics, Inc. and its affiliated companies. All rights reserved. IB-WTRBOTA

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