# HOMEDICS

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Instruction Manual and  
Warranty Information

BK-250

**2 year**  
limited warranty

## IMPORTANT SAFEGUARDS WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

### READ ALL INSTRUCTIONS BEFORE USING DANGER- TO REDUCE THE RISK OF ELECTROCUTION:

- ALWAYS unplug the appliance from the electrical outlet immediately after using and before cleaning.
- DO NOT reach for an appliance that has fallen into water. Unplug it immediately.
- DO NOT use while bathing or in shower.
- DO NOT place or store appliance where it can fall or be pulled into a tub or sink. DO NOT place in or drop into water or other liquid.
- NEVER use pins or other metallic fasteners with this appliance.
- Keep dry - DO NOT operate in a wet or moist condition.

### WARNING - TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS:

- An appliance should NEVER be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. DO NOT use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- Keep cord away from heated surfaces.
- NEVER use while sleeping or fall asleep while using the massager.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electrocution or injury to persons.
- DO NOT carry this appliance by supply cord or use cord as handle.
- To disconnect, turn all controls to the "off" position, then remove plug from outlet.
- This appliance is designed for personal, non-professional use only.
- DO NOT use outdoors.
- Temperatures sufficiently high to cause burns may occur regardless of the control setting. DO NOT use on an infant or invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the appliance frequently to reduce the risk of blistering.
- DO NOT crush. Avoid sharp folds.

### SAVE THESE INSTRUCTIONS

**Please Note:**

**For your safety, this product is equipped with a pressure activated switch to control the heat function. Heat will not operate unless pressure is applied to this switch, located on the seat bottom of the unit. Once pressure is removed, heat will turn off. After applying pressure, please allow 5-10 minutes for heat to reach maximum temperature.**

## CAUTION — PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- NEVER leave the appliance unattended, especially if children are present.
- NEVER cover the appliance when it is in operation.
- DO NOT use this product for more than 15 minutes at a time.
- Extensive use could lead to the product's excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- NEVER use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bed. The massage has a stimulating effect and can delay sleep.
- NEVER use this product while in bed.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This unit should not be used by children or invalids without adult supervision.

**Please Note:**

Not recommended for use on wooden surfaces as the zipper may damage the wood. Caution is also recommended when using on upholstered furniture.

**Caution:  
All servicing of this massager must be performed by authorized HoMedics service personnel only.**

## Instructions for Use

**For Use in Home or Office**

Plug adaptor into a 120 volt AC outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach massager to any chair for a luxurious massage as you read, rest or even work. See pages 6 - 7 for instructions on the use of the hand control.

**For Use in Your Car**

Simply plug the 12 volt auto adaptor into a cigarette outlet and plug the receptacle end into the input jack, located along the side of the cushion. Slip the straps over your seat to enjoy a relaxing massage as you drive. See pages 6 - 7 for instructions on the use of the hand control.

# BackComfort

## 5-Motor Back Cushion

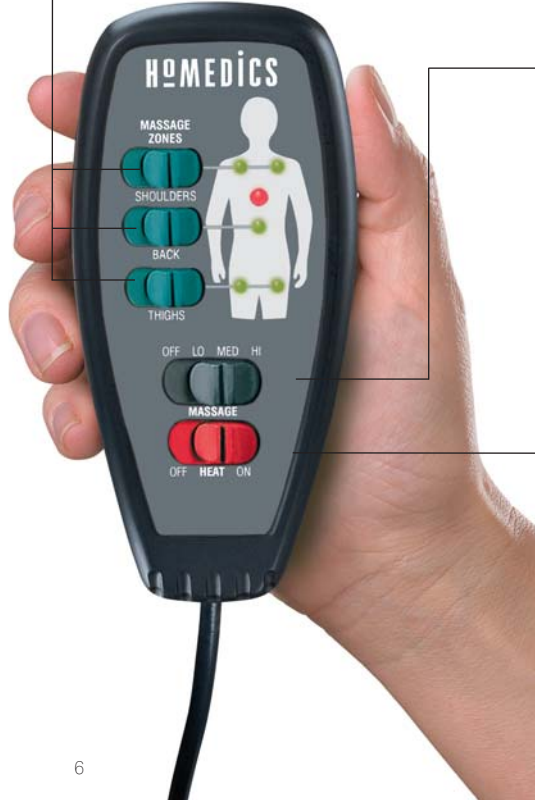
### Targeted Massage Buttons

Zone buttons activate motors to massage specific areas: shoulders, lower back and thighs. Customize your massage by choosing one specific area, any combination of areas, or all three. To select a massage area, simply slide the button to the right and the corresponding L.E.D. light will illuminate. To deselect, simply slide the button to the left and the corresponding L.E.D. lights will turn off.

### Power Adaptors

This massager is powered by a custom 120 volt AC home adaptor, and 12 volt DC auto adaptor (cigarette lighter plug-in style). Both are included with your unit.

**NOTE:** When selecting the heat feature with massaging action, there will be a slight decrease in the strength of the massage output. This adjustment is perfectly normal and should not be interpreted as a defect.

6

### On/Off

Before connecting power source, be sure ON/OFF controls are in the OFF position. Select LO, MED, or HI to activate continuous massage mode.

### Heat

Soothing, penetrating heat is activated by sliding this button to the right. To turn off heat function, simply slide the button to the left. A glowing L.E.D. light indicates that heat is on.

7

# HoMedics

## TWO YEAR LIMITED WARRANTY (Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

### Mail To:

**HoMedics**  
**Service Center, Dept. 168**  
**3000 Pontiac Trail**  
**Commerce Township,**  
**MI 48390**

**e-mail:**  
**[cservice@homedics.com](mailto:cservice@homedics.com)**

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit:  
[www.homedics.com](http://www.homedics.com)

©2003 HoMedics, Inc. and its affiliated companies. all rights reserved.  
HoMedics® is a registered trademark of HoMedics, Inc. and its affiliated companies.  
The beauty of living well.™ is a trademark of HoMedics, Inc. and its affiliated companies. All rights reserved.  
IB-BK250A

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