# HOMEDICS™

#1 BRAND IN MASSAGE\*

PLEASE TAKE A  
MOMENT NOW  
REGISTER YOUR PRODUCT AT:

[www.homedics.com/register](http://www.homedics.com/register)

Your valuable input regarding this  
product will help us create the  
products you will want in the future.

# SHIATSU MASSAGING CHAISE LOUNGER

INSTRUCTION MANUAL AND WARRANTY INFORMATION

2-YEAR LIMITED WARRANTY

MCS-1220H

# IMPORTANT SAFETY INSTRUCTIONS

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

READ ALL INSTRUCTIONS BEFORE USING.

## DANGER — TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Never use pins or other metallic fasteners with this appliance.

## WARNING — TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK, FIRE, OR INJURY TO PERSONS:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- NEVER operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the appliance to HoMedics Service Center for examination and repair.
- Keep the cord away from heated surfaces.
- NEVER drop or insert any object into any opening.
- DO NOT operate where aerosol (spray) products are being used or where oxygen is being administered.
- DO NOT operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- DO NOT carry this appliance by the supply cord or use cord as a handle.
- To disconnect, turn all controls to the “off” position, then remove plug from outlet.
- DO NOT use outdoors.
- NEVER operate the appliance with the air openings blocked. Keep the air openings free of lint, hair, and the like.

- Use heated surfaces carefully. May cause serious burns. Do not use over insensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.
- This appliance is designed for personal, non-professional use.
- DO NOT use this massager in close proximity to loose clothing or jewelry.
- Keep long hair away from massager while in use.

## SAVE THESE INSTRUCTIONS

### CAUTION — PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- This product is not intended for medical use. It is intended only to provide a luxurious massage.
- Consult your doctor prior to using this product if:
  - You are pregnant
  - You have a pacemaker
  - You have any concerns regarding your health
- Not recommended for use by diabetics.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 15 minutes at a time.
- Extensive use could lead to the product’s excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- DO NOT use this product as a substitute for medical attention.
- DO NOT use this product before bedtime. The massage has a stimulating effect and can delay sleep.
- This product should NEVER be used by any individual suffering from any physical ailment that would limit the user’s capacity to operate the controls or who has sensory deficiencies in the lower half of their body.
- This appliance is intended for household use only.

## INSTRUCTIONS FOR USE

1. Assemble the chaise according to the set-up instructions included in the box with your unit.
2. Remove the adapter from its box and plug the receptacle into the input jack located on the left side of the chaise.
3. Plug into an AC 120-volt outlet.
4. Once your chaise is set up, you can start enjoying the shiatsu massage mechanism according to the directions on page 8-9.
5. You may choose to lift the fabric flap and let it hang behind the seat back for a more intense massage.
6. You can change the reclining position of your chaise by folding the back of the chaise forward completely, then pushing it back away from the seat until you hear the hinge click. This is the most reclined position. To sit up straighter, lift the back of the chaise forward slightly until you hear another click. Repeat this process until your desired recline angle is reached. There are 4 total recline positions to enjoy.
7. To change the position of the leg rest, first pull up on the end of the leg rest until you hear the hinge click. Next, push the end of the leg rest toward the floor until you hear a second click. To move the leg rest up, lift the leg rest slightly until you hear it click into place. There are 3 leg rest positions you can enjoy.
8. Your chaise can also be utilized for seating without the massage mechanism activated.

---

**Caution:** To avoid pinching do not lean on the back massage mechanism in the chaise when adjusting your body position.

**Do not jam or force any part of your body in the moving massage mechanism.**

**Note:** There is a 15-minute auto shut off on the unit for your safety.

## MAINTENANCE

### TO STORE

Place massage chaise in its box or in a safe, cool, dry place. Avoid contact with sharp edges or pointed objects that might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit.

### TO CLEAN

Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean.
- Do not attempt to repair the chaise. There are no user-serviceable parts. For service, call the Consumer Relations telephone number listed in the Warranty section.

---

**NOTE:** Only gentle force should be exerted against the unit in order to eliminate any/all risk of injury. You may soften massage force by placing a towel between yourself and the unit.

## FCC STATEMENT

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

The actual product materials and fabrics may vary from the picture shown.

## POWER BUTTON

To turn on the massage functions, first press the power button. The massage chaise will power on. Next, you can choose the full, upper, or lower back massage zone with optional neck massage.

## NECK MASSAGE

To turn on neck massage mechanism, press the "NECK MASSAGE" button.

## NECK MASSAGE HEIGHT

You may adjust the neck massage shiatsu nodes by holding down either "HEIGHT" button until the desired position is reached. The up arrow will move the mechanism upward, and the down arrow will move it downward. Once you release the button, the mechanism will stop in that place.

## BACK SPOT MESSAGE

While the unit is in kneading massage, press one of the "SPOT" button arrows to stop and focus the shiatsu action in one place. Once the "SPOT" function is activated, press the up arrow to move the massage mechanism up, or press the down arrow to move it downward.

## SPEED

While the unit is in kneading massage, press the "H/L" button to adjust the speed that the massage nodes will travel. The default speed is high. Pressing the "H/L" button once will decrease the speed to low.

## DEMO BUTTON

For a brief demonstration of the features of the massage chaise, press the "DEMO" button. The unit will briefly go through each function. Once completed, the unit automatically turns off. The LED light will blink to indicate active function.

## NECK MASSAGE WITH SOOTHING HEAT

To turn on soothing heat in the neck massage nodes, press the "HEAT" button while the neck massage is in use. To turn off heat, press the button again.

## BACK MESSAGE ZONES

You can select full, upper, or lower back massage. To select a zone, press the button, and the LED will illuminate. To deselect, press the button again or select a different zone.

## BACK MESSAGE WITH SOOTHING HEAT

To turn on soothing heat in the back massage nodes, press the "HEAT" button while the back massage is in use. To turn off heat, press the button again.

Note: For your safety, heat can only be turned on while a massage program is running.

## 2-YEAR LIMITED WARRANTY

HoMedics sells its products with the intent that they are free of defects in manufacture and workmanship for a period of 2 years from the date of original purchase, except as noted below. HoMedics warrants that its products will be free of defects in material and workmanship under normal use and service. This warranty extends only to consumers and does not extend to retailers.

To obtain warranty service on your HoMedics product, contact a Consumer Relations representative for assistance. Please make sure to have the model number of the product available.

HoMedics does not authorize anyone, including but not limited to retailers, the subsequent consumer purchaser of the product from a retailer, or remote purchasers, to obligate HoMedics in any way beyond the terms set forth herein. This warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; improper installation; unauthorized repairs or modifications; improper use of electrical/power supply; loss of power; dropped product; malfunction or damage of an operating part from failure to provide manufacturer's recommended maintenance; transportation damage; theft; neglect; vandalism; or environmental conditions; loss of use during the period the product is at a repair facility or otherwise awaiting parts or repair; or any other conditions whatsoever that are beyond the control of HoMedics.

This warranty is effective only if the product is purchased and operated in the country in which the product is purchased. A product that requires modifications or adoption to enable it to operate in any other country than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under this warranty.

THE WARRANTY PROVIDED HEREIN SHALL BE THE SOLE AND EXCLUSIVE WARRANTY. THERE SHALL BE NO OTHER WARRANTIES EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS OR ANY OTHER OBLIGATION ON THE PART OF THE COMPANY WITH RESPECT TO PRODUCTS COVERED BY THIS WARRANTY. HOMEDICS SHALL HAVE NO LIABILITY FOR ANY INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES. IN NO EVENT SHALL THIS WARRANTY REQUIRE MORE THAN THE REPAIR OR REPLACEMENT OF ANY PART OR PARTS THAT ARE FOUND TO BE DEFECTIVE WITHIN THE EFFECTIVE PERIOD OF THE WARRANTY. NO REFUNDS WILL BE GIVEN. IF REPLACEMENT PARTS FOR DEFECTIVE MATERIALS ARE NOT AVAILABLE, HOMEDICS RESERVES THE RIGHT TO MAKE PRODUCT SUBSTITUTIONS IN LIEU OF REPAIR OR REPLACEMENT.

This warranty does not extend to the purchase of opened, used, repaired, repackaged, and/or resealed products, including but not limited to sale of such products on internet auction sites and/or sales of such products by surplus or bulk resellers. Any and all warranties or guarantees shall immediately cease and terminate as to any products or parts thereof that are repaired, replaced, altered, or modified, without the prior express and written consent of HoMedics.

This warranty provides you with specific legal rights. You may have additional rights that may vary from state to state and country to country. Because of individual state and country regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit [www.homedics.com](http://www.homedics.com). For Canada, please visit [www.homedics.ca](http://www.homedics.ca).

### FOR SERVICE IN THE USA:

Email: [cservice@homedics.com](mailto:cservice@homedics.com)

Monday–Friday 8:30am–7:00pm EST

1.800.466.3342

### FOR SERVICE IN CANADA:

Email: [cservice@homedicsgroup.ca](mailto:cservice@homedicsgroup.ca)

Monday–Friday 8:30am–5:00pm EST

1.888.225.7378

\*Source: The NPD Group. January 2017–December 2017, Retail Tracking Service.  
©2018 HoMedics, LLC. All rights reserved.

HoMedics is a registered trademark of HoMedics, LLC. HoMedics #1 Brand in Massage is a trademark of HoMedics, LLC. Distributed by Homedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390  
IB-MCS1220H

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©2018 HoMedics, LLC. Todos los derechos reservados.
HoMedics es una marca registrada y HoMedics #1 Brand In Massage es una marca comercial de HoMedics, LLC.
Distribuido por HoMedics, LLC, 3000 N Pontiac Trail, Commerce Township, MI 48390