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Consumer Relations  
 HoMedics  
 Service Center Dept. 168  
 3000 Pontiac Trail  
 Commerce Township, MI  
 48390

Correo electrónico:  
[cservice@homedics.com](mailto:cservice@homedics.com)

**HOMEDICS®**  
**THERAPIST SELECT**  
 The beauty of living well.™

## Programmable Percussion Massager with Heat

### Instruction Manual and Warranty Information

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**HoMEDICS**<sup>®</sup>  
THERAPIST SELECT

## IMPORTANT SAFETY INSTRUCTIONS:

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

### DANGER

-To reduce the risk of electric shock:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Do not use while bathing or showering.
- Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.

### WARNING

- To reduce the risk of burns, fire, electric shock or injury to persons:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- Never block the air openings of the appliance or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep air openings free of lint, hair, etc.
- Never use while sleeping or fall asleep while using the massager.
- Never drop or insert any object into any opening.

2

**Caution:**  
**In case of pregnancy, diabetes or illness, consult your doctor before using massager.**

**Caution:**  
**All servicing of this unit must be performed by authorized HoMedics service personnel only.**

- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not use on or near eyes or other highly sensitive areas.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electrical shock or injury to persons.
- Do not carry this appliance by supply cord or use cord as handle.
- Do not use the vibrating massager directly on one concentrated area of the body. Vibrating massagers are designed to be used in a continuous motion, constantly changing the massage area.
- To disconnect, turn all controls to the "OFF" position, then remove plug from outlet.
- This appliance is designed for household use only.

## SAVE THESE INSTRUCTIONS

**Caution – Please read all instructions carefully before operating.**

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 20 minutes at a time.
- Extensive use could lead to excessive heating and a shorter product life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- Do not use this product before bed. The massage has a stimulating effect and can delay sleep.
- Never use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children without adult supervision.

3

## Instructions for Use

This appliance has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

1. The Programmable Percussion Massager is powered by and comes with a UL-approved cord. Untie cord strap, extend cord and insert plug into 120-volt household electrical outlet to power unit.
2. To turn unit on, press POWER button located above foam-cushioned handle. A corresponding LED light will illuminate to indicate all selections. (See Figure 1.)
3. Choose the desired massage option: steady or program.
4. For steady massage, press MASSAGE button. To increase or decrease the massage intensity level, press the up or down arrows on each side of the PROGRAM button. The intensity level is indicated by the LED lights in the center of the program panel.
5. For a programmed massage, press the PROGRAM button on the upper left side. Choose any of the three program options:
  - P1 provides slow, steady percussion action to relax and revitalize.
  - P2 provides intense massage action to penetrate and soothe.
  - P3 provides short, quick bursts for energizing stimulation.
 An interval level may be chosen by pressing the INTERVAL button on the upper right side. There are three interval options: I1 (low), I2 (moderate) and I3 (high).
6. Gently apply massage heads to the area you wish to treat, and move slowly in a circular motion.

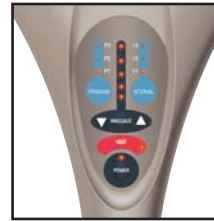
Figure 1

7. Treat the target area for a short period of time, and then move to another area. Never treat the same area longer than 3 minutes. **CAUTION:** As stated in the "IMPORTANT SAFETY INSTRUCTIONS" section of this manual, never use this appliance on or near the eyes, temples or other highly sensitive areas.
8. A massage session of longer than 20 minutes is not recommended.
9. When treatment is completed, turn the unit off and unplug the power supply. Store the unit properly in a cool, dry place, preferably in its original packaging.

## To Change Attachments

Your massager comes with 4 custom massage head attachments. Each set is a different firmness:

- Hard (light grey)
- Medium (medium grey)
- Soft (black)
- Heat (red)

To remove the massage heads from the unit, rotate each counter-clockwise. Rotate clockwise to fasten the massage heads. (See Figure 3.)

Figure 3

## Programmable Percussion Massager with Heat

Manual or program massage options

3 Program massage functions

5 Massage intensities

Optional heat to soothe tired muscles

3 Different interval levels

Ergonomic, contoured design

Dual pivoting massage heads (3,100 pulses per minute)

Extra Long foam-cushioned handle and cord for easy reach & maneuverability

4 Custom massage attachment sets included: Heat, soft, medium and hard

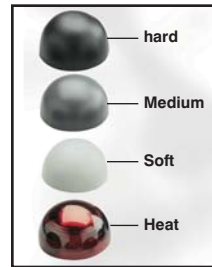
Figure 2

### Massage with Thermal Heat

1. Locate the two red (heat) attachments. Attach to the massage head by turning clockwise to secure.
2. Choose the desired massage option – steady or programmed – and press the HEAT button.
3. The heat feature will work only in conjunction with the heat massage attachments. The heat indicator will illuminate when used with other attachments, but will not be felt. (See Figure 2.)

**NOTE: To eliminate the risk of being burned, never use the heat function without the heat attachments. Do not use the thermal heat continuously for more than 20 minutes. Do not use the thermal heat if you have sensitive skin, poor circulation, discolored areas, swelling, burning, inflammation, or where skin eruptions or sores are present. Heat should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your physician.**

### Application

The massager may be used while standing, sitting or lying on a firm surface. The long handle provides comfortable, sufficient reach for self-massage application.

### Maintenance

#### To Clean

- Be sure to unplug the unit and allow it to cool before cleaning. Use a soft, slightly damp cloth to wipe. Never allow water or any liquids to come into contact with the handle.

#### To Store

- Unplug the appliance from the outlet and allow it to cool before storing in its box or a clean dry place. Coil the cord and secure it with the supplied fastener. Do not hang the unit by the powercord.

**TWO YEAR LIMITED WARRANTY**

(Valid in USA only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of original purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling.

Upon receipt, HoMedics will repair or replace, as appropriate, your product and return it to you, postpaid. If it is appropriate to replace your product, HoMedics will replace the product with the same product or a comparable product at HoMedics' option. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

For more information regarding our product line in the USA, please visit: [www.homedics.com](http://www.homedics.com)

**Mail To:**  
Consumer Relations  
HoMedics  
Service Center-Dept. 168  
3000 Pontiac Trail  
Commerce Township MI  
48390

**Email:**  
[cservice@homedics.com](mailto:cservice@homedics.com)

©2005 HoMedics, Inc. and its affiliated companies, all rights reserved.  
HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies.  
TherapistSelect™ is a trademark of HoMedics Inc. and its affiliated companies.  
IB-PA4H

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