

**Therapist
Select™**

Two Year Limited Warranty

(Valid in USA Only)

HoMedics, Inc., guarantees this product free from defects in material and workmanship for a period of two years from the date of purchase, except as noted below.

This HoMedics product warranty does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorized accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of HoMedics. This warranty is effective only if the product is purchased and operated in the USA. A product that requires modification or adaptation to enable it to operate in any county other than the country for which it was designed, manufactured, approved and/or authorized, or repair of products damaged by these modifications is not covered under warranty. HoMedics shall not be responsible for any type of incidental, consequential or special damages. All implied warranties, including but not limited to those implied warranties of fitness and merchantability, are limited in the total duration of two years from the original purchase date.

To obtain warranty service on your HoMedics product, either hand deliver or mail the unit and your dated sales receipt (as proof of purchase), postpaid, along with a certified check or money order in the amount of \$5.00 payable to HoMedics, Inc. to cover handling. Upon receipt, HoMedics will repair or replace, as appropriate, your unit and return it to you, postpaid. Warranty is solely through HoMedics Service Center. Service of this product by anyone other than HoMedics Service Center voids warranty.

This warranty provides you with specific legal rights. You may have additional rights which may vary from state to state. Because of individual state regulations, some of the above limitations and exclusions may not apply to you.

**For more information regarding our product line in the USA,
please visit: www.homedics.com**

Mail To:

**Consumer Relations
HoMedics
Service Center, Dept. 168
3000 Pontiac Trail,
Commerce Township, MI
48390**

**Email:
cservice@homedics.com**

©2001 HoMedics, Inc. and its affiliated companies, all rights reserved.
HoMedics® is a registered trademark of HoMedics Inc. and its affiliated companies.
Therapist Select™ is a trademark of HoMedics Inc. and its affiliated companies.
All rights reserved.
IB-PAB100

**2 year
limited warranty**

**Therapist
Select™**

TherapistSelect™ Percussion Action Back Massager



**Instruction Manual and
Warranty Information**

PA-B100

**2 year
limited warranty**

IMPORTANT SAFETY INSTRUCTIONS:

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

DANGER

—To reduce the risk of electric shock:

- Always unplug the appliance from the electrical outlet immediately after using and before cleaning.
- Do not reach for an appliance that has fallen into water. Unplug it immediately.
- Do not use while bathing or showering.
- Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.

WARNING

—To reduce the risk of burns, fire, electric shock or injury to persons:

- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use and before putting on or taking off parts or attachments.
- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics; specifically any attachments not provided with the unit.
- Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return it to HoMedics Service Center for examination and repair.
- Keep cord away from heated surfaces.
- Never use while sleeping or fall asleep while using the massager.

- Never drop or insert any object into any opening.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electrical shock or injury to persons.
- Do not carry this appliance by supply cord or use cord as handle.
- To disconnect, turn all controls to the “OFF” position, then remove plug from outlet.
- This appliance is designed for household use only.

SAVE THESE INSTRUCTIONS

Caution — Please read all instructions carefully before operating.

- If you have any concerns regarding your health consult your doctor before using this product.
- Individuals with pacemakers should consult a doctor before use.
- Never leave the appliance unattended, especially if children are present.
- Never cover the appliance when it is in operation.
- Do not use this product for more than 20 minutes at a time.
- Extensive use could lead to the product’s excessive heating and shorter life. Should this occur, discontinue use and allow the unit to cool before operating.
- Never use this product directly on swollen or inflamed areas or skin eruptions.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.

- Do not use this product before bed. The massage has a stimulating effect and can delay sleep.
- Never use this product while in bed.
- This product should never be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- This unit should not be used by children without adult supervision.

Instructions for Use

For Use in Home or Office

Plug adapter into a 120 volt AC outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach Therapist Select to your chair by attaching the adjustable straps together in back of the chair for a luxurious massage as you read, rest or even work.

For Use in Your Car

Simply plug the 12 volt auto adapter into a cigarette outlet and plug the receptacle end into the input jack, located along the side of the cushion. Attach the adjustable straps around your seat to enjoy a relaxing massage as you drive.

Strapping System

This massager incorporates a strapping system which allows you to fasten it to almost any chair or auto seat. Just pull the straps around the seat or chair to fit properly, then attach the adjustable straps together. Your massager won't slip or slide away.

Caution:
All servicing of this massager must be performed by authorized HoMedics Service Personnel only.

Maintenance

To Store: Place massager in its box or a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, do not wrap the power cord around the unit. Do not hang the unit by the controller cord.

To Clean: Unplug the unit and allow it to cool before cleaning. Clean only with a soft, slightly damp sponge. Never allow water or any other liquids to come into contact with the unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish or paint thinner to clean.
- Do not attempt to repair this unit. There are no user serviceable parts. For service, send the unit to the HoMedics Service Center address listed in the warranty section.

Massage away tension and aches.

Thank you for purchasing the Therapist Select percussion action back massager by HoMedics. This, like all of HoMedics products, is built with high-quality craftsmanship and is guaranteed to provide you soothing massage relief for years to come.

HoMedics is dedicated to bringing you better health through innovation. However, you should never substitute this, or any other manufacturer's massager, for professional medical attention. Persistent ailments and symptoms could indicate a more serious condition.

TherapistSelect™

Percussion Action Back Massager

Easy-to-Use Controller

Allows you to customize the massage intensity.

On/Off Massage Strength

Before connecting power source, be sure ON/OFF control is in the OFF position. Therapist Select gives you variable intensity of massage strengths. To change the strength of your massage, simply turn the dial clockwise to increase the intensity of the massage. Turn the dial counter-clockwise to decrease the intensity of the massage.

Power Adapters

Therapist Select is powered by a custom 120 volt AC home adapter and 12 volt DC auto adapter (cigarette lighter plug-in style). Both are included with your unit.

Use only these supplied adapters. If you happen to lose an adapter and require a replacement, please contact HoMedics Consumer Relations.

On/Off
Indicator



Ergonomic Design

Helps provide support to your back.



Dual Pivoting Heads
For an invigorating massage.

Adjustable Strapping System

Fastens the massager to most chairs and auto seats.