## TRIUMPH™ VIBRATION PINPOINT MASSAGER

INSTRUCTION BOOKLET AND SAFETY INFORMATION

SR-PPM

### FIND MORE HELP ONLINE

Visit the URL to the right to find how-to photos, usage tips, and additional information about this product.

[www.homedics.com/triumph](http://www.homedics.com/triumph)

### INSTRUCTIONS FOR USE

- Remove the battery cover by turning to the unlock position. Insert 2 “C” batteries (not included) into the compartment (Fig. 1). Replace the cover and turn back to the lock position.
- Press the power button once to activate massager.
- To rotate the massaging head from pinpoint (A) to multi-point (B), pull the massage head out and rotate clockwise, then push it down to lock it into place. (Fig. 2)
- After completing your massage, press the power button to turn the massager off.

FIG 1

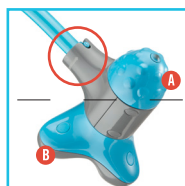
FIG 2

### TO CLEAN

Clean only with a soft fabric cloth. Never allow water or other liquids to come into contact with this unit.

- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, or paint thinner to clean.

## IMPORTANT SAFETY INSTRUCTIONS

WHEN USING THIS APPLIANCE, BASIC PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:  
READ ALL INSTRUCTIONS BEFORE USING

- This product is not intended for medical use.
- Close supervision is necessary when this appliance is used by, on, or near children, invalids, or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use attachments not recommended by HoMedics, specifically any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- Not for use by children.
- Turn off appliance when not in use.
- Never operate this appliance if it has been damaged or if it is not working properly. Return it to the HoMedics service center for examination and repair.
- Keep away from heated surfaces.

## SAVE THESE INSTRUCTIONS

CAUTION: PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE OPERATING.

- If you have concerns regarding your health, consult your doctor before using this product.
- Those with the following conditions should consult their doctor before using this product:
  - Pregnancy
  - Malignant tumor
  - Diabetes
  - Conditions requiring rest
  - Heart disease
  - Pacemakers

- If at any time soreness, rash, or pain occurs, discontinue use and consult your physician.
- NEVER use this product directly on swollen or inflamed areas, skin eruptions, or an open wound.
- A massage session longer than 20 minutes is not recommended.
- Attention California Residents:

**WARNING:** This product contains chemicals known to the State of California to cause cancer, birth defects, and other reproductive harm.

## BATTERY PRECAUTIONS

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of batteries may cause damage to the unit.
- Do not mix old and new batteries.
- Do not mix alkaline, standard (carbon-zinc) or rechargeable (ni-cad, ni-mh, etc.) batteries.
- Do not dispose of batteries in fire. Batteries may explode or leak.
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.
- Clean the battery contacts and also those of the device prior to battery installation.
- Remove depleted batteries promptly.
- Dispose of properly in accordance with all state, province, and country regulations.

This product contains small parts that may present a choking hazard for young children.

Caution: All servicing of this massager must be performed by authorized HoMedics Service Personnel only.

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